

# INSTRUCTION SHEET

## ENDOSKELETAL COMPONENT

### 4-Hole Fastener Kit / Part No. 4HFK 14

#### (4) M6 x 14mm Fasteners

These M6 x 14mm fasteners are for use with non-rotatable adapters. Use of longer fasteners may be required for rotatable adapters.

FASTENER SELECTION	POTENTIAL RESULT
Thread length too short	Inadequate thread engagement
Thread length too long	Bottoming-out or binding

### CAUTIONS

- Apply Loctite® 242® on all fastener threads.
- Torque evenly to 12 N·m (8.8 ft·lbs).
- Do not expose this product to corrosive materials.

*Failure to follow these guidelines may result in one or all of the following: noise, movement, damage, and catastrophic failure. For further questions, contact College Park Technical Service.*



# college park

800.728.7950 | 586.294.7950 | college-park.com



**COLLEGE PARK INDUSTRIES**

27955 College Park Dr. Warren, MI 48088 USA



**EMERGO EUROPE**

Prinsessegracht 20, 2514 AP The Hague, Netherlands

**Australian Sponsor EMERGO AUSTRALIA**

Level 20, Tower II, Darling Park, 201 Sussex Street  
Sydney, NSW 2000 Australia



MADE IN THE USA

122 INS4HFK 14 170804

©2017 College Park Industries, Inc. All rights reserved. College Park is a registered trademark of College Park Industries, Inc.