

INSTRUCTION SHEET

ENDOSKELETAL COMPONENT

4-Hole Fastener Kit / Part No. 4HFK 16

(4) M6 x 16mm Fasteners

These M6 x 16mm fasteners are for use with rotatable adapters. Use of shorter fasteners may be required for non-rotatable adapters.

FASTENER SELECTION	POTENTIAL RESULT
Thread length too short	Inadequate thread engagement
Thread length too long	Bottoming-out or binding

CAUTIONS

- Apply Loctite® 242® on all fastener threads.
- Torque evenly to 12 N·m (8.8 ft-lbs).
- Do not expose this product to corrosive materials.

Failure to follow these guidelines may result in one or all of the following: noise, movement, damage, and catastrophic failure. For further questions, contact College Park Technical Service.



college park

800.728.7950 | 586.294.7950 | college-park.com



COLLEGE PARK INDUSTRIES

27955 College Park Dr. Warren, MI 48088 USA



EMERGO EUROPE

Prinsessegracht 20, 2514 AP The Hague, Netherlands

Australian Sponsor EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex Street
Sydney, NSW 2000 Australia



MADE IN THE USA
196 INS 4HFK 16170804

©2017 College Park Industries, Inc. All rights reserved. College Park is a registered trademark of College Park Industries, Inc.