

INSTRUCTION SHEET

ENDOSKELETAL COMPONENT

4-Hole Pyramid Adapter, Diagonal Offset / Part No. 4HPA DO T

TECHNICAL INFORMATION

Material	Titanium
Build Height	7 mm (0.28 in)
Diagonal Offset	7 mm (0.28 in)
Assembly Weight	64 g
Patient Weight Limit	136 kg (300 lbs)
Warranty	3 Year, limited
Torque (Fasteners)	12 N·m (8.8 ft-lbs)

The M6 x 14mm fasteners included are for use with non-rotatable adapters. Use of longer fasteners may be required for rotatable adapters.

FASTENER SELECTION	POTENTIAL RESULT
Thread length too short	Inadequate thread engagement
Thread length too long	Bottoming-out or binding

- ⚠ CAUTIONS**
- Apply Loctite® 242® on all fastener threads.
 - Torque evenly to specified settings.
 - Do not expose this product to corrosive materials.

Failure to follow these guidelines may result in one or all of the following: noise, movement, damage, and catastrophic failure. For further questions, please contact College Park Technical Service.



college park

800.728.7950 | 586.294.7950 | college-park.com



COLLEGE PARK INDUSTRIES
27955 College Park Dr.
Warren, MI 48088 USA



EMERGO EUROPE
Prinsessegracht 20, 2514 AP
The Hague, Netherlands

MADE IN THE USA
223 INS 4HPA DO 170804



Australian Sponsor
EMERGO AUSTRALIA
Level 20, Tower II, Darling Park,
201 Sussex Street,
Sydney, NSW 2000 Australia

©2017 College Park Industries, Inc. All rights reserved. College Park is a registered trademark of College Park Industries, Inc.