

# INSTRUCTION SHEET

## ENDOSKELETAL COMPONENT

### 4-Hole Pyramid Adapter, SS / Part No. 4HPA S

#### TECHNICAL INFORMATION

<b>Material</b>	Stainless Steel
<b>Build Height</b>	7 mm (0.28 in)
<b>Assembly Weight</b>	75 g
<b>Patient Weight Limit</b>	181 kg (400 lbs)
<b>Warranty</b>	3 Year, limited
<b>Torque</b> (Fasteners)	12 N·m (8.8 ft-lbs)

The M6 x 14mm fasteners included are for use with non-rotatable adapters. Use of longer fasteners may be required for rotatable adapters.

FASTENER SELECTION	POTENTIAL RESULT
Thread length too short	Inadequate thread engagement
Thread length too long	Bottoming-out or binding

- ⚠ CAUTIONS**
- Apply Loctite® 242® on all fastener threads.
  - Torque evenly to specified settings.
  - Do not expose this product to corrosive materials.

*Failure to follow these guidelines may result in one or all of the following: noise, movement, damage, and catastrophic failure. For further questions, please contact College Park Technical Service.*



**college park**

800.728.7950 | 586.294.7950 | college-park.com



**COLLEGE PARK INDUSTRIES**  
27955 College Park Dr.  
Warren, MI 48088 USA



**EMERGO EUROPE**  
Prinsessegracht 20, 2514 AP  
The Hague, Netherlands

**MADE IN THE USA**

442 INS 4HPA S 170804



**Australian Sponsor**  
**EMERGO AUSTRALIA**  
Level 20, Tower II, Darling Park,  
201 Sussex Street,  
Sydney, NSW 2000 Australia

©2017 College Park Industries, Inc. All rights reserved. College Park is a registered trademark of College Park Industries, Inc.