INSTRUCTION SHEET | ENDOSKELETAL COMPONENTS

4-Hole Pyramid Adapters



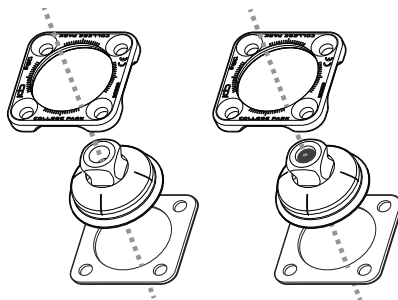
FIGURE 1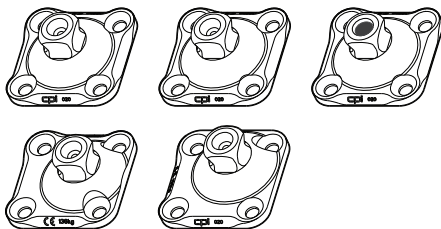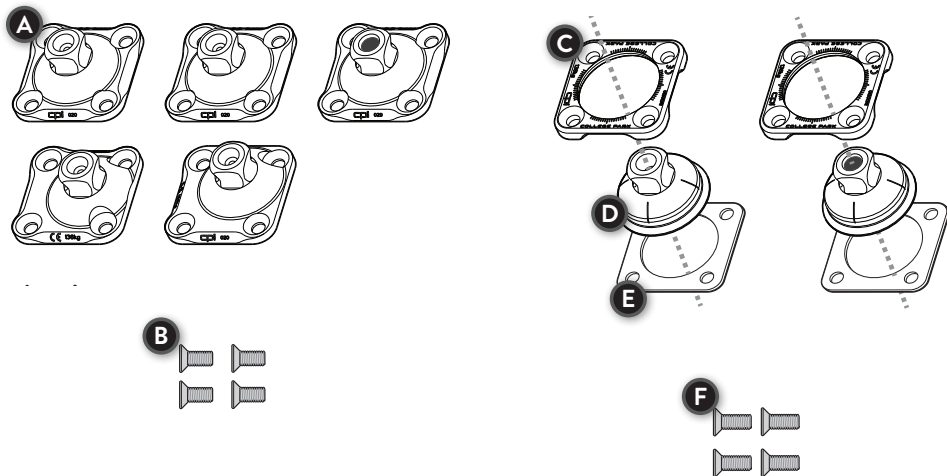

INSTRUCTION SHEET | ENDOSKELETAL COMPONENTS

4-Hole Pyramid Adapters

PRODUCT DESCRIPTION

This endoskeletal component consists of a 4-hole pyramid adapter (non-rotatable or rotatable). Each includes four mounting fasteners. Rotatable adapters are constructed with a rotatable base, rotatable pyramid, and retainer plate.

KEY COMPONENTS *(Figure 1)*

NON-ROTATABLE ADAPTERS

A. 4-Hole Pyramid Adapter

- Standard (Ti) ▪ Standard (SS)
- Standard w/ Pin-Lock Hole ▪ Axial Offset
- Diagonal Offset

B. (4) M6 x 14mm Mounting Fasteners

ROTATABLE ADAPTERS

C. Rotatable Base

D. Pyramid

- Rotatable w/ Pin-Lock Hole ▪ Rotatable

E. Retainer Plate

F. (4) M6 x 16mm Mounting Fasteners

This diagram is to help familiarize you with the unique parts of a 4-hole pyramid adapter, an endoskeletal component. These parts are referenced in the instructions and used when speaking with a technical service representative..

INTENDED USE

Endoskeletal Components are intended for aligning and attaching feet, ankles, and knees on lower limb prosthesis users.

INDICATIONS:

- Lower limb amputations

CONTRAINDICATIONS:

- None known

WARNING

- Apply Loctite® 242® on all mounting fastener threads.
- Torque evenly to specified settings.
- Do not expose this product to corrosive materials.

Failure to follow these guidelines may result in one or all of the following: noise, movement, damage, and catastrophic failure. For further questions, please contact College Park Technical Service.

WARRANTY INSPECTION/ MAINTENANCE INFORMATION

College Park recommends that you schedule your patients for check-ups according to the Warranty Inspection schedule below.

High patient weight and/or impact level may require more frequent inspections. Prosthetic components should be inspected after the first 30 days of use. We recommend you visually inspect the entire prosthesis for excessive wear and fatigue at each consultation.

WARRANTY INSPECTION SCHEDULE FOR ENDOSKELETAL COMPONENTS: 30 DAYS, THEN ANNUALLY.

TECHNICAL SPECIFICATIONS

	4-Hole Pyramid Adapter, Standard, Ti	4-Hole Pyramid Adapter w/ Pin-Lock Hole, Standard, Ti	4-Hole Pyramid Adapter, Rotatable, Ti	4-Hole Pyramid Adapter w/ Pin-Lock Hole, Rotatable, Ti	4-Hole Pyramid Adapter, Standard, SS	4-Hole Pyramid Adapter, Axial Offset, Ti	4-Hole Pyramid Adapter, Diagonal Offset, Ti
Part No.	4HPA T	4HPH T	4HPAR T	4HPHR T	4HPA S	4HPA AO T	4HPA DO T
Material	Titanium	Titanium	Titanium	Titanium	Stainless Steel	Titanium	Titanium
Build Height	7 mm (0.28 in)	7 mm (0.28 in)	9.5 mm (0.37 in)	9.5 mm (0.37 in)	7 mm (0.28 in)	7 mm (0.28 in)	7 mm (0.28 in)
Offset	--	--	--	--	--	7 mm (0.28 in)	7 mm (0.28 in)
Assembly Weight	46 g	44 g	77 g	74 g	75 g	55 g	64 g
Patient Weight Limit	136 kg (300 lbs)	136 kg (300 lbs)	136 kg (300 lbs)	136 kg (300 lbs)	181 kg (400 lbs)	136 kg (300 lbs)	136 kg (300 lbs)
Warranty	3 Year, limited	3 Year, limited	3 Year, limited	3 Year, limited	3 Year, limited	3 Year, limited	3 Year, limited
Torque (Fasteners)	12 N·m (8.8 ft-lbs)	12 N·m (8.8 ft-lbs)	12 N·m (8.8 ft-lbs)	12 N·m (8.8 ft-lbs)	12 N·m (8.8 ft-lbs)	12 N·m (8.8 ft-lbs)	12 N·m (8.8 ft-lbs)

GUIDELINES

Non-Rotatable Adapters

M6 x 14mm fasteners included are for use with non-rotatable adapters.

Use of longer fasteners may be required for rotatable adapters.

Rotatable Adapters

M6 x 16mm fasteners are for use with College Park rotatable adapters.

Use of shorter fasteners may be required for non-rotatable adapters.

FASTENER SELECTION	POTENTIAL RESULT
Thread length too short	Inadequate thread engagement
Thread length too long	Bottoming-out or binding

TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 am – 5:30 pm (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.

LIABILITY

The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer

**CAUTION**

College Park products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards are achieved only when College Park products are used with other recommended College Park components. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients.

**CAUTION**

If any problems occur with the use of this product, immediately contact your medical professional. The prosthetist and/or patient should report any serious incident* that has occurred in relation to the device to College Park Industries, Inc. and the competent authority of the Member State in which the prosthetist and/or patient is established.

*“Serious incident” is defined as any incident that directly or indirectly led, may have led, or might lead to any of the following: (a) the death of a patient, user, or other person, (b) the temporary or permanent serious deterioration of a patient's, user's, or other person's state of health, (c) a serious public health threat.

COMPLIANCE

This device has been tested according to ISO 10328 standard to three million load cycles.

Depending on patient activity this may correspond to 3-5 years of use.

ISO 10328 - LABEL

PART NUMBER	WEIGHT LIMIT (KG)	LABEL TEXT
4HPA T	136	ISO 10328-P7-136 kg
4HPAR T		
4HPH T		
4HPHR T		
4HPA DO T		
4HPA AO T	181	ISO 10328-P8-181 kg
4HPR S		

ISO 10328 - “P” - “m”kg*) 

**) Body mass limit not to be exceeded!*

For specific conditions and limitations of use, see intended use section of manufacturer's written instructions.

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©2025 College Park Industries, Inc. All rights reserved.