

INSTRUCTION SHEET

ENDOSKELETAL COMPONENT

4-Hole Pyramid Adapter, Ti, Rotatable / Part No. 4HPAR T

TECHNICAL INFORMATION

Material	Titanium
Build Height	9.5 mm (0.37 in)
Assembly Weight	77 g
Patient Weight Limit	136 kg (300 lbs)
Warranty	3 Year, limited
Torque (Fasteners)	12 N·m (8.8 ft-lbs)

The M6 x 16mm fasteners are for use with College Park rotatable adapters. Use of shorter fasteners may be required for non-rotatable adapters.

FASTENER SELECTION	POTENTIAL RESULT
Thread length too short	Inadequate thread engagement
Thread length too long	Bottoming-out or binding

- ⚠ CAUTIONS**
- Apply Loctite® 242® on all fastener threads.
 - Torque evenly to specified settings.
 - Do not expose this product to corrosive materials.

For further questions, please contact College Park Technical Service.



college park

800.728.7950 | 586.294.7950 | college-park.com



COLLEGE PARK INDUSTRIES
27955 College Park Dr.
Warren, MI 48088 USA



EMERGO EUROPE
Prinsessegracht 20, 2514 AP
The Hague, Netherlands

MADE IN THE USA 
220 INS 4HPAR 170804

Australian Sponsor
EMERGO AUSTRALIA
Level 20, Tower II, Darling Park,
201 Sussex Street,
Sydney, NSW 2000 Australia

©2017 College Park Industries, Inc. All rights reserved. College Park is a registered trademark of College Park Industries, Inc.