

INSTRUCTION SHEET

ENDOSKELETAL COMPONENT

4-Hole Pyramid Adapter with pin-lock hole / Part No. 4HPH T

TECHNICAL INFORMATION

Material	Titanium
Build Height	7 mm (0.28 in)
Assembly Weight	44 g
Patient Weight Limit	136 kg (300 lbs)
Warranty	3 Year, limited
Torque (Fasteners)	12 N·m (8.8 ft-lbs)

The M6 x 14mm fasteners included are for use with non-rotatable adapters.
Use of longer fasteners may be required for rotatable adapters.

FASTENER SELECTION	POTENTIAL RESULT
Thread length too short	Inadequate thread engagement
Thread length too long	Bottoming-out or binding

- ⚠ CAUTIONS**
- Apply Loctite® 242® on all fastener threads.
 - Torque evenly to specified settings.
 - Do not expose this product to corrosive materials.

Failure to follow these guidelines may result in one or all of the following: noise, movement, damage, and catastrophic failure. For further questions, please contact College Park Technical Service.



college park

800.728.7950 | 586.294.7950 | college-park.com



COLLEGE PARK INDUSTRIES, INC.
27955 College Park Dr.
Warren, MI 48088 USA



EMERGO EUROPE
Prinsessegracht 20, 2514 AP
The Hague, Netherlands

MADE IN THE USA



372 INS 4HPH 180604

Australian Sponsor
EMERGO AUSTRALIA
Level 20, Tower II, Darling Park,
201 Sussex Street,
Sydney, NSW 2000 Australia

©2018 College Park Industries, Inc. All rights reserved. College Park is a registered trademark of College Park Industries, Inc.