

# INSTRUCTION SHEET

## ENDOSKELETAL COMPONENT

### 4-Hole Pyramid Receiver, Rotatable, Ti / Part No. 4HPRR T

#### TECHNICAL INFORMATION

<b>Material</b>	Titanium
<b>Build Height</b>	19.1 mm (0.75 in)
<b>Assembly Weight</b>	90 g
<b>Patient Weight Limit</b>	136 kg (300 lbs)
<b>Warranty</b>	3 Year, limited
<b>Torque (Fasteners)</b>	12 N·m (8.8 ft-lbs)
<b>Torque (Set Screws)</b>	15 N·m (11 ft-lbs)

The M6 x 16mm fasteners are for use with College Park rotatable adapters. Use of shorter fasteners may be required for non-rotatable adapters.

#### FASTENER SELECTION

#### POTENTIAL RESULT

Thread length too short	Inadequate thread engagement
Thread length too long	Bottoming-out or binding

- ⚠ CAUTIONS**
- Apply Loctite® 242® on all M6 x16mm fastener threads.
  - M8 x 14mm set screws do not require Loctite.
  - Torque evenly to specified settings.
  - Do not expose this product to corrosive materials.

For further questions, please contact College Park Technical Service.



## college park

800.728.7950 | 586.294.7950 | college-park.com



**COLLEGE PARK INDUSTRIES**

27955 College Park Dr.  
Warren, MI 48088 USA



**EMERGO EUROPE**

Prinsessegracht 20, 2514 AP  
The Hague, Netherlands

**MADE IN THE USA**



221 INS 4HPRR 170804

**Australian Sponsor**

**EMERGO AUSTRALIA**

Level 20, Tower II, Darling Park,  
201 Sussex Street,  
Sydney, NSW 2000 Australia

©2017 College Park Industries, Inc. All rights reserved. College Park is a registered trademark of College Park Industries, Inc.