# INSTRUCTION SHEET | ENDOSKELETAL COMPONENTS

## Double Ended Adapters


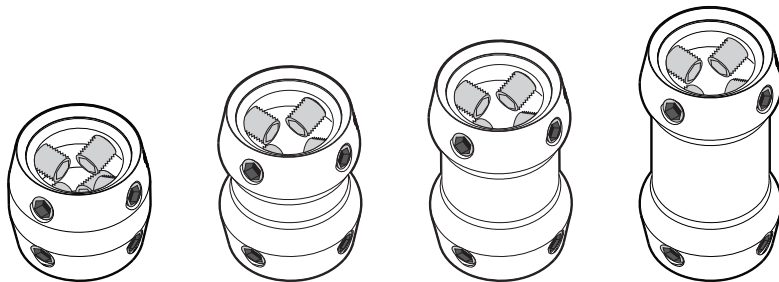
FIGURE 1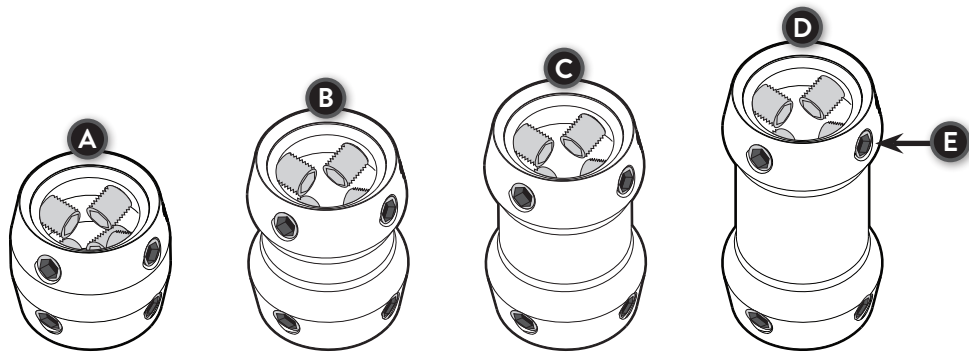

# INSTRUCTION SHEET | ENDOSKELETAL COMPONENTS

## Double Ended Adapters

### PRODUCT DESCRIPTION

This endoskeletal component consists of a double ended adapter. It includes eight set screws.

### KEY COMPONENTS *(Figure 1)*

#### Double Ended Adapter

- A. 35mm      B. 45mm  
C. 60mm      D. 75mm

#### E. (4) M8 x 14mm Set Screws

*This diagram is to help familiarize you with the unique parts of a double ended adapter, an endoskeletal component. These parts are referenced in the instructions and used when speaking with a technical service representative.*

### INTENDED USE

Endoskeletal Components are intended for aligning and attaching feet, ankles, and knees on lower limb prosthesis users.

### INDICATIONS:

- Lower limb amputations

### CONTRAINDICATIONS:

- None known

### WARNING

- Torque evenly to specified settings.
- Do not expose this product to corrosive materials.

*Failure to follow these guidelines may result in one or all of the following: noise, movement, damage, and catastrophic failure. For further questions, please contact College Park Technical Service.*

## WARRANTY INSPECTION/ MAINTENANCE INFORMATION

College Park recommends that you schedule your patients for check-ups according to the Warranty Inspection schedule below.

*High patient weight and/or impact level may require more frequent inspections. Prosthetic components should be inspected after the first 30 days of use. We recommend you visually inspect the entire prosthesis for excessive wear and fatigue at each consultation.*

WARRANTY INSPECTION SCHEDULE FOR ENDOSKELETAL COMPONENTS: 30 DAYS, THEN ANNUALLY.

### TECHNICAL SPECIFICATIONS

	Double Ended Adapter, Ti, 35mm	Double Ended Adapter, Ti, 45mm	Double Ended Adapter, Ti, 60mm	Double Ended Adapter, Ti, 75mm
<b>Part No.</b>	<b>DR T35</b>	<b>DR T45</b>	<b>DR T60</b>	<b>DR T75</b>
<b>Material</b>	Titanium	Titanium	Titanium	Titanium
<b>Build Height</b>	35 mm (1.38 in)	45 mm (1.77 in)	60 mm (2.36 in)	75 mm (2.95 in)
<b>Assembly Weight</b>	95 g	110 g	123 g	136 g
<b>Patient Weight Limit</b>	136 kg (300 lbs)	136 kg (300 lbs)	136 kg (300 lbs)	136 kg (300 lbs)
<b>Warranty</b>	3 Year, limited	3 Year, limited	3 Year, limited	3 Year, limited
<b>Torque (Set Screws)</b>	15 N·m (11 ft-lbs)	15 N·m (11 ft-lbs)	15 N·m (11 ft-lbs)	15 N·m (11 ft-lbs)
<b>Torque (Set Screws)</b>	15 N·m (11 ft-lbs)	15 N·m (11 ft-lbs)	15 N·m (11 ft-lbs)	
<b>Torque (Clamp Screw)</b>	10 N·m (7.4 ft-lbs)	6 N·m (4.4 ft-lbs)	6 N·m (4.4 ft-lbs)	

### GUIDELINES

M8 x 14mm set screws with nylon patch are included with the adapter. Loctite® is not required. M8 x 14mm set screws with nylon patch are included with the adapter. Loctite® is not required.

## TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 am – 5:30 pm (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.

## LIABILITY

The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer

### CAUTION

College Park products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards are achieved only when College Park products are used with other recommended College Park components. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients.

### CAUTION

If any problems occur with the use of this product, immediately contact your medical professional. The prosthetist and/or patient should report any serious incident\* that has occurred in relation to the device to College Park Industries, Inc. and the competent authority of the Member State in which the prosthetist and/or patient is established.

\*'Serious incident' is defined as any incident that directly or indirectly led, may have led, or might lead to any of the following; (a) the death of a patient, user, or other person, (b) the temporary or permanent serious deterioration of a patient's, user's, or other person's state of health, (c) a serious public health threat.

## COMPLIANCE

This device has been tested according to ISO 10328 standard to three million load cycles.

Depending on patient activity this may correspond to 3-5 years of use.

### ISO 10328 - LABEL

PART NUMBER	WEIGHT LIMIT (KG)	LABEL TEXT
DR T35		
DR T45	136	ISO 10328-P7-136 kg
DR T60		
DR T75		

### ISO 10328 - "P" - "m"kg\*)

\*) *Body mass limit not to be exceeded!*

*For specific conditions and limitations of use, see intended use section of manufacturer's written instructions.*

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©2023 College Park Industries, Inc. All rights reserved.