

INSTRUCTION SHEET

ENDOSKELETAL COMPONENT

25 mm Tube Clamp with Adult Receiver, Ti / Part No. TC T 2530

TECHNICAL INFORMATION

Material	Titanium
Build Height	45 mm (1.77 in)
Assembly Weight	83 g
Patient Weight Limit	75 kg (165 lbs)
Warranty	3 Year, limited
Torque (Set Screws)	15 N·m (11 ft-lbs)
Torque (Clamp Screw)	10 N·m (7 ft-lbs)

M6 x 12mm set screws with nylon patch and M5 x 14mm clamp screw are included with the adapter; no Loctite® required.

- ⚠ CAUTIONS**
- Torque evenly to specified settings.
 - Do not expose this product to corrosive materials.

For further questions, please contact College Park Technical Service.



college park

800.728.7950 | 586.294.7950 | college-park.com

MADE IN THE USA

452 INS TC T 2530 170804



COLLEGE PARK INDUSTRIES

27955 College Park Dr.
Warren, MI 48088 USA



EMERGO EUROPE

Prinsessegracht 20, 2514 AP
The Hague, Netherlands

Australian Sponsor

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park,
201 Sussex Street,
Sydney, NSW 2000 Australia

©2017 College Park Industries, Inc. All rights reserved. College Park is a registered trademark of College Park Industries, Inc.