

INSTRUCTION SHEET

C100 CYLINDRICAL SHUTTLE LOCK COMPONENT

4-Hole Fastener Kit / Part No. 4HFK 25

(4) M6 x 25mm Fasteners

These M6 x 25mm fasteners are for use with the C100 Cylindrical Shuttle Lock. Always consider the thickness of the mounting surface and adapter when selecting fasteners.

| FASTENER SELECTION | POTENTIAL RESULT |
|-------------------------|------------------------------|
| Thread length too short | Inadequate thread engagement |
| Thread length too long | Bottoming-out or binding |

CAUTIONS

- Apply Loctite® 242® on all fastener threads.
- Torque evenly to 12 N·m (8.8 ft·lbs).
- Do not expose this product to corrosive materials.

Failure to follow these guidelines may result in one or all of the following: noise, movement, damage, and catastrophic failure. For further questions, contact College Park Technical Service.



college park

800.728.7950 | 586.294.7950 | college-park.com



COLLEGE PARK INDUSTRIES, INC

27955 College Park Dr. Warren, MI 48088 USA



EMERGO EUROPE

Prinsessegracht 20, 2514 AP The Hague, Netherlands

Australian Sponsor EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex Street
Sydney, NSW 2000 Australia



MADE IN THE USA
689 INS 4HFK 25 171005

©2017 College Park Industries, Inc. All rights reserved. College Park is a registered trademark of College Park Industries, Inc.