STRIDE STUDIO

for ODYSSEY iQ

APP INSTRUCTIONS

cpi-ifu.com

Bulgarski • Čeština • Dansk • Deutsch • Español • Suomen Kieli
Français • Hrvatski • Magyar • Italiano • Nihongo • Kugo • Norsk
Polski • Română • Slovenčina • Slovensko • Svenska • Türkçe • Zhōngwén

Paper copies are available free of charge upon request via cpi-ifu.com and QR code above.

college park

TECHNOLOGY for the HUMAN RACE

FIGURE 1

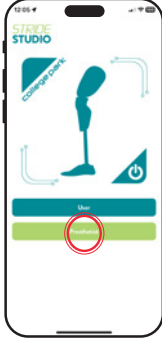
FIGURE 2

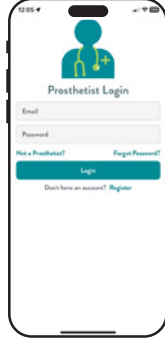
FIGURE 3

FIGURE 4

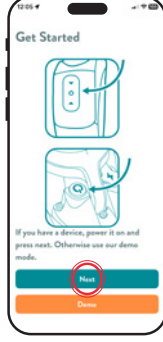
FIGURE 5

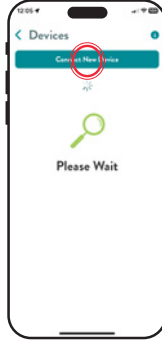
FIGURE 6

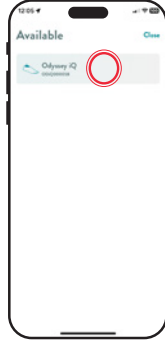
FIGURE 7

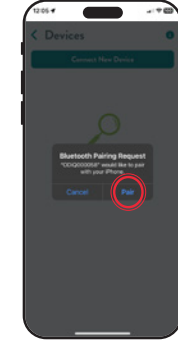
FIGURE 8

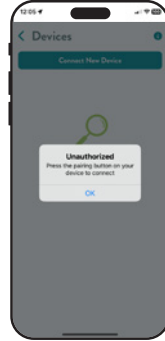
FIGURE 9

FIGURE 10

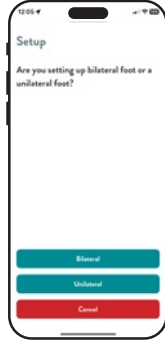
FIGURE 11

FIGURE 12

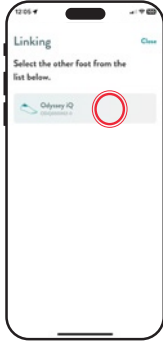
FIGURE 13

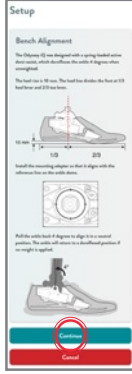
FIGURE 14

FIGURE 15

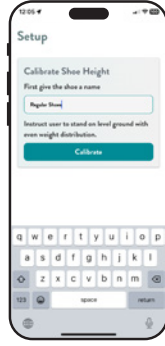
FIGURE 16

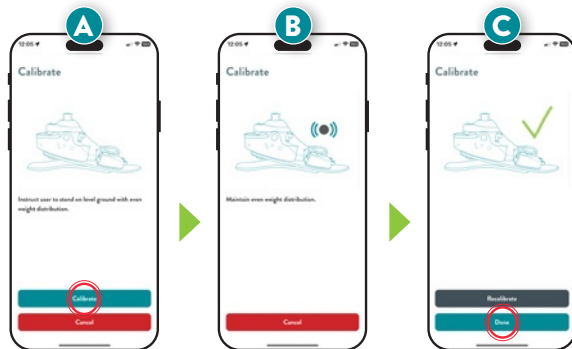
FIGURE 17

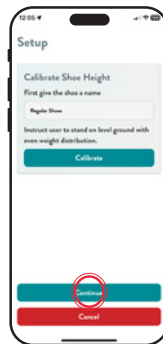
FIGURE 18

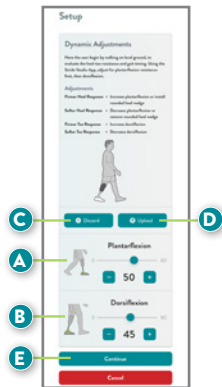
FIGURE 19

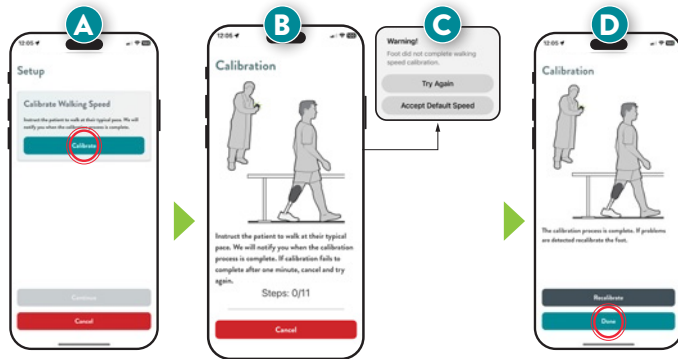
FIGURE 20

FIGURE 21

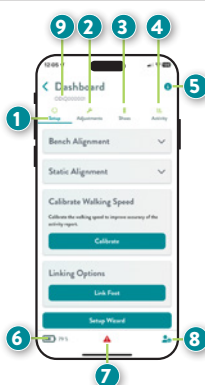
FIGURE 22

FIGURE 23

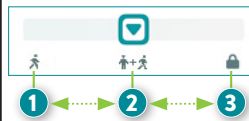
FIGURE 24

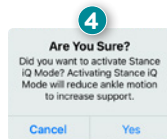
FIGURE 25

FIGURE 26

FIGURE 27

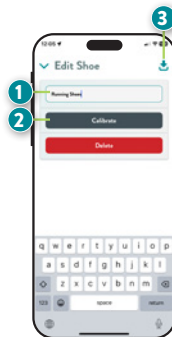
FIGURE 28

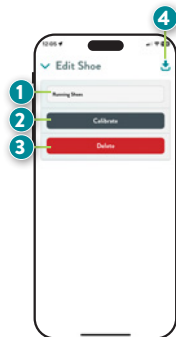
FIGURE 29

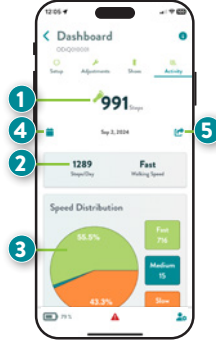
FIGURE 30

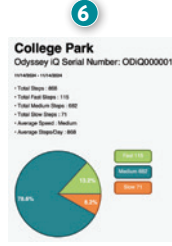
FIGURE 31

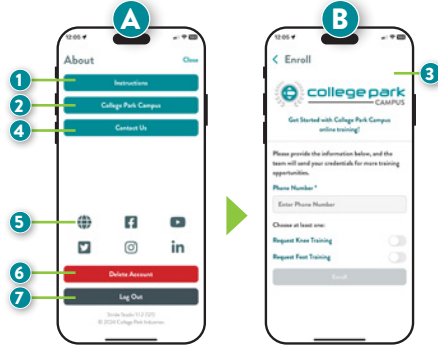
FIGURE 32


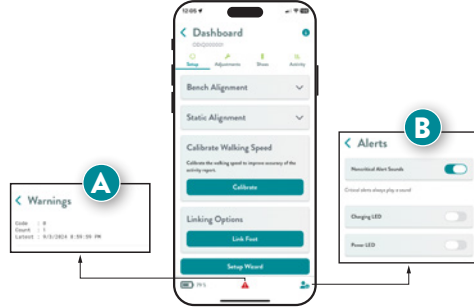
FIGURE 33

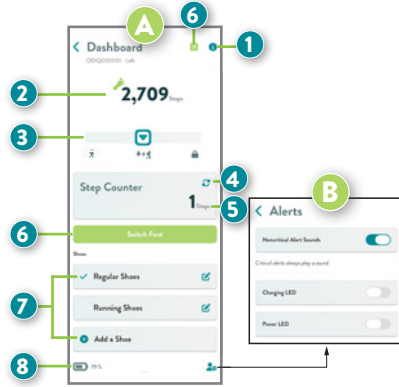LANGUAGES

EN-English	8
BG-Bulgarian	17
CS-Čeština	26
DA-Dansk	35
DE-Deutsch	44
ES-Español	53
FI-Suomen Kieli	62
FR-Français	71
HR-Hrvatski	80
HU-Magyar	89
IT-Italiano	98
JA-Nihongo	107
KO-Kugo	116
NO-Norsk	125
PL-Polski	134
RO-Română	143
SK-Slovenčina	152
SL-Slovensko	161
SV-Svenska	170
TR-Türkçe	179
ZH-Zhōngwén	188

APP SETUP

DOWNLOAD STRIDE STUDIO APP

College Park's Stride Studio App is available for both iOS & Android devices. Download Stride Studio from Apple App Store or Google Play Store.

OPERATING SYSTEMS SUPPORTED:

- Apple iOS
- Android

Select Stride Studio icon on the device to launch the app.

LANDING SCREEN

The landing screen allows you to select whether you are the user or prosthetist (*Figure 1*).

- Users: There is no login required, Select “User” and go to Stride Studio App for Users section..
- Prosthetists: Select “Prosthetist” and you will be taken to the Prosthetist Login Screen (*Figure 2*).

PROSTHETIST LOGIN

When using the app for the first time, prosthetists must register for an account. Select “Register”.

Register for an account by completing the mobile registration form in the app (*Figures 3-A, 3-B*).

PAIR DEVICE SCREEN

ENABLE BLUETOOTH

Go to Settings on your phone or tablet, then tap Bluetooth.

Launch the Stride Studio app and log in. The app will prompt you to allow it access to Bluetooth. You must grant it access. You will land on the “Get Started” screen. Select “Next” to pair the Icon knee (*Figure 4*).

“Demo” mode allows you to view the settings without pairing to a foot.

BLUETOOTH PAIRING

The foot must be paired to the Stride Studio app when it is used for the first time. Select “Connect New Device” (Figure 5), and a list of available devices will be displayed. Select the serial number of the foot from the menu (Figure 6). A Bluetooth pairing request will pop up. Press the “Pair” button (Figure 7). The app will display an “Unauthorized” pop up. Press the power button on the front of the ankle for 1 second to complete the pairing process (Figure 8).

Note: The foot will be identified by the Serial Number, which is located on the left side of the ankle.

Note: Phone or tablet must be closer than 33 feet (10 meters) of the Odyssey iQ foot.

Note: Only one device may be paired to the foot at a time. Prosthetists must disconnect from the foot before the user can pair in the app. Disconnect the foot from the “Devices” screen or by closing the app.

For Bilateral Users: To program easy switching between left and right feet, follow the Linking process in the Setup Wizard.

TRAINING VIDEO

If this is your first time setting up an Icon knee, the app will play a mandatory training video (Figure 8). This is a brief overview of the physical setup and programming of the foot. It is also accessible later through College Park Campus.

Note: This video will only play the first time the prosthetist sets up a knee.

SETUP WIZARD

If this is the first time setting up a new Odyssey iQ foot, the Setup Wizard will automatically launch. The Setup Wizard is designed to guide the prosthetist through the required steps of the setup process.

Order of Operation:

- Linking
- Bench & Static Alignment
- Calibrate Shoe Height
- Dynamic Adjustments
- Calibrate Walking Speed

LINKING

The Linking process is set up by the prosthetist, so that prosthetists and bilateral users can easily switch between the left foot and right foot during app use.

Begin by selecting whether the setup is for a “Unilateral” (Figure 10-A) or “Bilateral” (Figure 10-B) foot. If the foot is unilateral, the app will skip the linking process and advance to the next section.

If bilateral, select “Left Foot” or “Right Foot” to identify the foot that is currently paired with the app (Figure 11).

To link the other foot, select it from the list (Figure 12) and it will be assigned to the remaining side.

BENCH & STATIC ALIGNMENT

Displays the recommended bench (Figure 13) and static alignment for the Odyssey iQ foot (Figure 14). Select “Continue” to move to the next section.

CALIBRATE SHOE HEIGHT

This section calibrates the angle for shoes with heel heights between 0-0.80 in (0-2.0 cm).

To calibrate the shoe height, users should begin in a standing position, wearing their preferred shoes. Enter a name for the shoe (Figure 15), then select “Calibrate”.

On the next screen, select “Calibrate” and follow the calibration process (Figures 16 A-C). When complete, select “Continue” to move to the next section (Figure 17).

DYNAMIC ADJUSTMENTS

Have the user begin by walking on level ground, to evaluate the heel-toe resistance and gait timing.

Use the sliders to adjust for plantarflexion resistance first (Figure 18-A), then dorsiflexion (Figure 18-B).

Tap the “Discard” button to undo any changes (Figure 18-C). The foot responds to resistance adjustments once they are uploaded (Figure 18-D).

Finalize dynamic alignment by observing the user walking on an inclining-declining surface (ramp). Make further adjustments to the resistance valves as necessary.

When complete, select “Continue” to move to the next section (Figure 18-E).

CALIBRATE WALKING SPEED

There is a calibration process to capture the user’s normal walking speed. This is to improve accuracy in foot performance.

To calibrate the walking speed, select “Calibrate” (Figure 19-A). Instruct the user to walk at their normal pace. A step counter will indicate when the user completes 11 “good” steps (Figure 19-B). If calibration was not successful, a warning will appear after 1 minute. You can either try again or accept the default speed (Figure 19-C). When the process is complete, select “Done” (Figure 19-D).

Select “Continue” to complete calibration and exit the Setup Wizard (Figure 20). Exiting the Setup Wizard will take you to the main dashboard.

DASHBOARD

The dashboard screen is the main hub of the Odyssey iQ foot setup. It provides access to all setup functions and critical information (Figure 21).

- 1 **Setup** - Alignment recommendations, calibrate walking speed, linking options, and setup wizard.
- 2 **Adjustments** - Adjustments to foot resistance and mode selection.
- 3 **Shoes** - Allows prosthetist and user to set custom shoe profiles.
- 4 **Activity** - Activity log: download and print reports.
- 5 **Information** - Technical instructions and online learning resources.
- 6 **Battery Status** - Indicates battery charge level of Odyssey iQ foot.
- 7 **Device Warnings** - List of error codes (if device experiences an error).
- 8 **Alerts Settings** - Turn non-critical alert sounds on/off.
- 9 **Serial Number** - Displays the serial number of the foot.

SETUP SCREEN (FIGURE 22)

- 1 **Alignment** - Bench and static alignment recommendations.
- 2 **Calibrate Walking Speed** - Captures normal walking speed to improve accuracy in foot performance.
- 3 **Linking Options** - Link or unlink left & right feet for easy switching during app use.
- 4 **Setup Wizard** - Step-by-step guide for the entire setup process.
- 5 **Switch Foot** - Switch between left & right feet. Icon will only be visible if linking is enabled.

ADJUSTMENTS SCREEN

MODE SELECTOR

Mode Selector – The slider bar allows you to toggle between walking mode (1) and Stance iQ mode (2), or from Stance iQ mode (2) to ankle lock (3) (Figure 23). An alert will appear each time the slider is moved to Stance iQ mode (4) (Figure 24).

RESISTANCE ADJUSTMENTS

The foot responds to resistance adjustments once they are uploaded (Figure 25).

Note: Toggle the foot into walking mode before making resistance adjustments.

- 1 **Plantarflexion** - Controls minimum (-) to maximum (+) resistance.
- 2 **Dorsiflexion** - Controls minimum (-) to maximum (+) resistance.
- 3 **Discard** - Undoes the change and reverts resistances back to the previous settings.
- 4 **Upload** - Saves the updated configuration to the foot.
- 5 **Reset to Factory Defaults** - Default settings to factory configuration.
- 6 **Stance iQ Timer** - Set the time it takes to enter Stance iQ mode (2-10 seconds).

SHOES SCREEN

CUSTOM SHOE PROFILES

This section calibrates the angle for shoes with heel heights between 0-0.80 in (0-2.0 cm). Assign a new shoe profile to optimize variance in heel heights or heel densities (very soft or very hard-heeled shoes). (Figure 26).

To change shoe profiles, users should begin in a standing position, wearing their preferred shoes. When a profile is selected, it will take effect immediately and remain active until a different profile is selected.

- 1 **Existing Mode** - Programmed modes are displayed here. A checked box indicates the current mode.
- 2 **Add a Shoe** - Click to add a new shoe profile.
- 3 **Edit** - Opens Edit screen.

ADD CUSTOM SHOE PROFILES

To add a new shoe profile, users should begin in a standing position, wearing their preferred shoes.

Select “Add a Shoe” from the “Shoes” screen (Figure 26). Enter a name for the new shoe, select “Calibrate” and follow the calibration process (Figure 27). When complete, tap the “Save” icon to apply the new shoe profile.

- 1 **Name** - Enter a name for the new shoe profile.
- 2 **Calibrate** - This section calibrates the angle and normal walking speed for shoes with heel heights between 0-0.80 in (0-2.0 cm).
Note: Bilateral shoes must be calibrated individually.
- 3 **Save** - Applies settings to shoe profile button.

EDIT CUSTOM SHOE PROFILES (FIGURE 28).

- 1 **Name** - Name of the selected shoe profile.
- 2 **Calibrate** - Re-calibrates the selected shoe.
- 3 **Delete** - Deletes the shoe profile.
- 4 **Save** - Applies changes to shoe profile button.

ACTIVITY SCREEN (FIGURES 29, 30)

- 1 **Step Counter** - Total step count for lifetime of device.
- 2 **Daily Average** - View average steps per day and average walking speed for the selected date range.
- 3 **Speed Distribution** - Visualizes walking speed distribution as pie chart.
- 4 **Activity Date Range** - Select date range of activity report.
- 5 **Export** - Generates a custom activity report as a PDF via text, email, etc. View up to 6 months of data.
- 6 **Activity Report** - Example of generated report.

ABOUT SCREEN

ABOUT & INFORMATION

This screen provides quick access to many resources (Figure 31).

- 1 **Technical Instructions** - Digital copy of the Odyssey iQ foot technical instructions.
- 2 **College Park Campus** - Online learning management system for product training.
- 3 **Enrollment Form** - Enroll in College Park Campus directly through the app.
- 4 **Contact Us** - Email College Park for support.
- 5 **Social Media** - Direct links to social media pages.
- 6 **Delete Account** - Deletes your login and password credentials.
- 7 **Log Out** - Log out of the Stride Studio App.

DEVICE WARNINGS / ALERT SETTINGS

DEVICE WARNINGS

If the foot issues an error code, it will be listed here. Information can be shared when communicating with Technical Support (Figure 32-A).

ALERT SETTINGS

Turn non-critical alert sounds on/off. Enable/disable Power LED and Charging LED. (Figure 32-B).

STRIDE STUDIO APP FOR USERS

STRIDE STUDIO APP FOR USERS

Users may choose to download the Stride Studio app, but it is not required.

PAIR DEVICE SCREEN

ENABLE BLUETOOTH

Go to Settings on your phone or tablet, then tap Bluetooth.

Launch the Stride Studio app and select “User”. The app will prompt you to allow it access to Bluetooth. You must grant it access. You will land on the “Devices” screen.

BLUETOOTH PAIRING

The foot must be paired to the Stride Studio app when it is used for the first time. Select “Connect New Device” (Figure 5), and a list of available devices will be displayed. Select the serial number of the foot from the menu (Figure 6). A Bluetooth pairing request will pop up. Press the “Pair” button (Figure 7). The app will display an “Unauthorized” pop up. Press the power button on the front of the ankle for 1 second to complete the pairing process (Figure 8).

Note: The foot will be identified by the Serial Number, which is located on the left side of the ankle.

Note: Phone or tablet must be closer than 33 feet (10 meters) of the Odyssey iQ foot.

Note: Only one device may be paired to the foot at a time.

For Bilateral Users: Your prosthetist can program easy switching between left and right feet during app use, see Switching Between Left and Right Feet section.

USER DASHBOARD

The user dashboard screen is the main hub of the Odyssey iQ foot (Figures 33-A, 33-B). It provides access to basic foot information, mode selection, and critical information.

1	About - Links to User Guide and social media resources.
2	Lifetime Step Counter - Total step count for lifetime of device.
3	Mode Selector - See Mode Selector section.
4	Reset - Resets the step counter to zero.
5	Step Counter - Tracks number of steps between resets.
6	Switch Foot - See Switching Between Left and Right Feet section.
7	Custom Shoe Profiles - See Custom Shoe Profiles section.
8	Battery Status - Indicates battery charge level of Odyssey iQ foot.
9	Alerts Settings - Turn non-critical alert sounds on/off. Enable/disable Power LED and Charging LED.

MODE SELECTOR

Mode Selector – The slider bar allows you to toggle between walking mode (1) and Stance iQ mode (2), or from Stance iQ mode (2) to ankle lock (3) (figure 23). An alert will appear each time the slider is moved to Stance iQ mode (4) (Figure 24).

SWITCHING BETWEEN LEFT AND RIGHT FEET

The foot linking process can be set up so that bilateral users can easily switch between their left and right feet during app use. This icon will only appear if linking has been enabled by your prosthetist (Figure 33).

Tap the “Switch Foot” icon to switch between left & right feet.

CUSTOM SHOE PROFILES

This section calibrates the angle for shoes with heel heights between 0-0.80 in (0-2.0 cm). Assign a new shoe profile to optimize variance in heel heights or heel densities (very soft or very hard-heeled shoes). (Figure 26).

To change shoe profiles, begin in a standing position, wearing your preferred shoes. When a profile is selected, it will take effect immediately and remain active until a different profile is selected.

- 1 **Existing Mode** - Programmed modes are displayed here. A checked box indicates the current mode.
- 2 **Add a Shoe** - Click to add a new shoe profile.
- 3 **Edit** - Opens **Edit** screen.

ADD CUSTOM SHOE PROFILES

To add a new shoe profile, begin in a standing position, wearing your preferred shoes.

Select “Add a Shoe” from the “Shoes” screen (Figure 26). Enter a name for the new shoe, select “Calibrate” and follow the calibration process (Figure 27). When complete, tap the “Save” icon to apply the new shoe profile.

- 1 **Name** - Enter a name for the new shoe profile.
- 2 **Calibrate** - This section calibrates the angle for shoes with heel heights between 0-0.80 in (0-2.0 cm).
Note: Bilateral shoes must be calibrated individually.
- 3 **Save** - Applies settings to shoe profile button.

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This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

MEDENVOY SWITZERLAND

Gotthardstrasse 28, 6302 Zug, Switzerland

MADE IN THE USA

©2026 College Park Industries, Inc. All rights reserved.

1173 INS SS ODIQ 260720

COLLEGE PARK INDUSTRIES, INC

27955 College Park Dr. Warren, MI 48088 USA

EMERGO EUROPE

Westervoortsedijk 60, 6827 AT Arnhem, The Netherlands

Australian Sponsor

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex Street,
Sydney, NSW 2000 Australia