


SIZING OPTIONS FOR HEEL WEDGE (Shown at actual size)

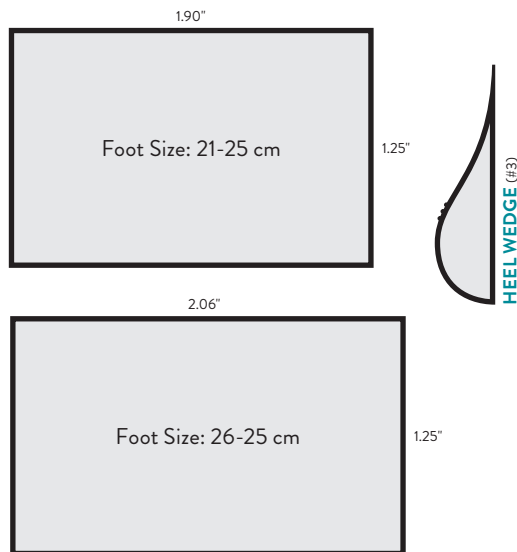

COLLEGE PARK INDUSTRIES
27955 College Park Dr.
Warren, MI 48088 USA


Australian Sponsor
EMERGO AUSTRALIA
Level 20, Tower II, Darling Park
201 Sussex Street
Sydney, NSW 2000 Australia

©2022 College Park Industries, Inc. All rights reserved.

EC REP **EMERGO EUROPE**
Prinsessegracht 20, 2514 AP
The Hague, Netherlands


CE **MADE IN THE USA**
1059 INS VY WK 220106

college park voyager™ HEEL WEDGE KIT

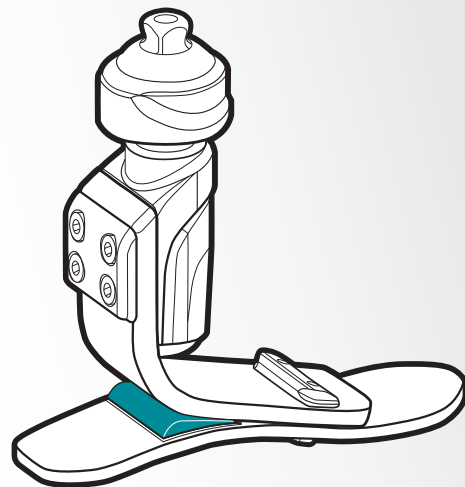


instruction
sheet

VOYAGER WEDGE PLACEMENT

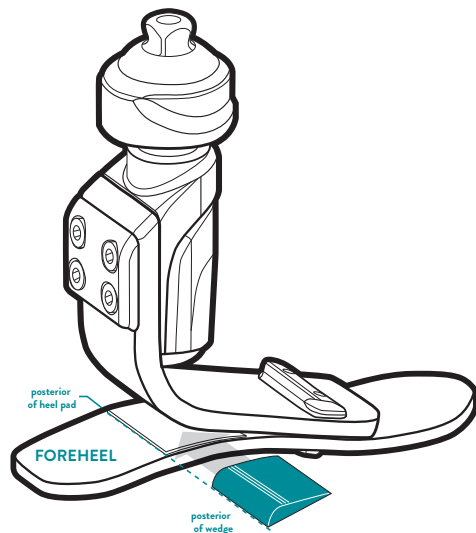

ADJUSTMENT OPTIONS FOR WEDGES

HEEL FIRMNESS	WEDGE #	INDICATORS
STANDARD	NONE	NONE
MEDIUM	1	I
FIRM	2	II
X-FIRM	3	III

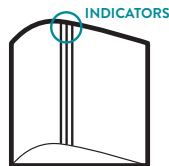

WEDGE INSTALLATION AND REMOVAL

SEMI-PERMANENT BOND (INITIAL FITTING)

- 1 Apply only the smallest amount of CA adhesive* (included) necessary to temporarily hold the wedge into position. Too much will inhibit removal.
- 2 When installing the wedge, align posterior edges of the wedge and pad.
- 3 Press on the wedge and hold it for 15 seconds.



WEDGE REMOVAL

To remove the rounded wedge without damage, pry it away with a finger or use a flat, blunt object such as a tongue depressor.

PERMANENT BOND (FINAL FITTING)

- 1 Apply a thin line of CA adhesive* around the bottom perimeter of the wedge.
- 2 When installing the wedge, align posterior edges of the wedge and pad.
- 3 Press on the wedge and hold it for 15 seconds.

NOTE: The wedge will NOT be removable once it has been bonded.



CAUTIONS

- Use only the correct size wedge in the foot (see back for sizing guide).
- Do not install more than one wedge into the heel position.
- Place wedge only in the recommended position.
- Do not force the wedge further forward into the toe spring.
- Do not substitute other objects in place of recommended wedges.

*Cyanoacrylate (CA) adhesive, included as part of the wedge kit.