metro liners®

GENESEE LINER

college park
TECHNOLOGY for the HUMAN RACE

technical instructions

التعليمات الفنية • Teknisk vejledning • Technische Anleitung
Τεχνικές οδηγίες • Instrucciones técnicas • Tekniset ohjeet
Instructions techniques • הוראות טכניות • Istruzioni tecniche
Technische instructies • Tekniske instruksjoner • Instrukcje techniczne
Instruções técnicas • Instruções técnicas • Технические инструкции •
Technické pokyny • Tekniska anvisningar • Teknik Talimatlar • 技术说明

FIGURE 1

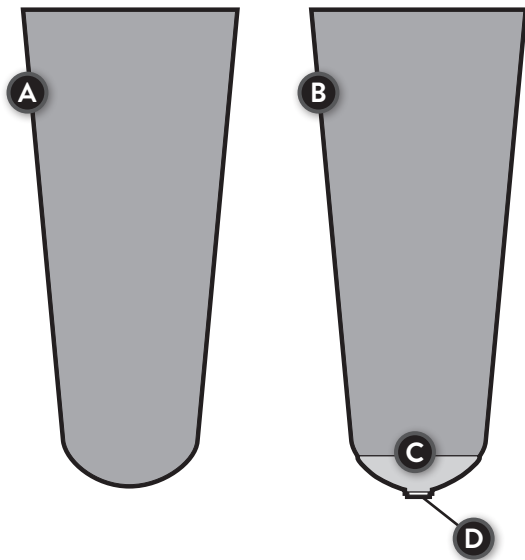
FIGURE 2


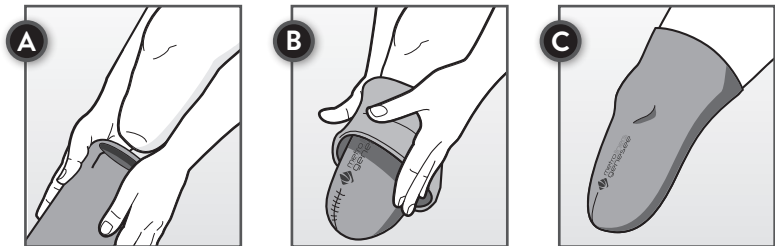
FIGURE 3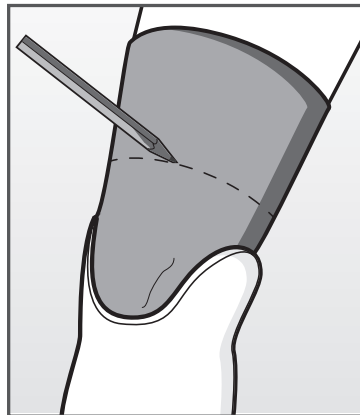

Package Contents

Genesee Gel Liner

Drying Stand (not shown)

Tools Recommended

Loctite® 242

EN

This diagram is to help familiarize you with the unique parts of the Genesee Gel Liner. These parts are referenced in the instructions and used when speaking with a technical service representative.

Key Components (Figure 1)

A. Cushion Liner

B. Locking Liner

C. Umbrella

D. Threaded Receiver

PRODUCT DESCRIPTION

The Genesee gel liner is a thermoplastic elastomer (TPE) gel liner, with fabric cover, that is available in cushion and locking options, two profiles, five sizes, and three thicknesses. It includes a drying stand.

INTENDED USE

The Genesee Liner is a flexible cover that is rolled onto a limb, which creates a barrier between the skin and prosthesis. Liners provide cushion and protection, and are often used as part of the suspension method for holding prosthesis onto a limb.

INDICATIONS:

- Lower limb amputations

CONTRAINDICATIONS:

- None known

TECHNICAL SPECIFICATIONS

Material	Thermoplastic Elastomer (TPE) Gel
Types	Cushion or Locking
Profiles	Uniform or Tapered
Sizes	M, M+, L, L+, XL
Thicknesses	3 mm, 6 mm, 9 mm
Overall Length	16.25 in (41.3 cm)

SIZE	M	M+	L	L+	XL
Proximal Circumference 15 in from distal end	9-13 in (23-33 cm)	13-17 in (33-43 cm)	14-18 in (36-46 cm)	16-21 in (41-53 cm)	17-27 in (43-69 cm)
Distal Circumference 1.5 in from distal end	8-11 in (20-28 cm)	8-11 in (20-28 cm)	9-12 in (23-30 cm)	11-15 in (28-38 cm)	12-20 in (30-51 cm)

LOCKING LINER INSTRUCTIONS

Locking liners consist of M10 x 1.5 threaded receivers for pin placement. Each liner must have its own pin installed. Use Loctite® 242® or equivalent to secure.

WARNING: Failure to properly secure pin to the liner may lead to noise, separation, and potential injury to your patient.

DONNING THE LINER (Figure 2)

- Invert the liner completely (gel facing out), and align the bottom of the liner to the distal end of limb, with the logo facing front.
- Roll liner onto limb.
- Ensure that air pockets have been eliminated and that there are no wrinkles in the liner.

FITTING GUIDELINES

HEAT STRETCHING

If necessary, the liner can be stretched over a positive model. Place in oven at 150°F (65°C), for 1-5 minutes. This can be repeated, but do not exceed 15 total minutes of heating. Allow liner to cool completely on model before removing and/or fitting on patient.

TRIMMING/ WAVE TRIMMERS (Figure 3)

The liner can be trimmed to fit each patient. With the patient wearing the liner, mark the liner at the desired location (not lower than 2" above socket trim line). Remove the liner from patient and cut using sharp scissors or equivalent.

Place one drop of CA adhesive on the stitched seam at the top edge of the liner, to prevent unraveling.

Use of a wave trimmer is recommended, and can alleviate skin irritation at the liner's edge.

PATIENT GUIDELINES

It is **required** that prosthetists review the User Guide with their patient. Failure to do so may result in skin irritation/injury to patient and reduced lifespan of the liner.

FABRICATION GUIDELINES

NEW VS. RETROFIT

It is recommended that you fit or replace Genesee Gel Liners to a socket that was fabricated using this liner.

Genesee Gel Liners are retro-fittable with OWW/Alpha® and other similar liners. Contact College Park with any questions or concerns.

CASTING CONSIDERATIONS

It is recommended to cast your patient while they are wearing the liner.

Use a barrier (such as plastic wrap, plastic bag, etc) to ease cast removal and prevent soiling of the liner.

Casting kits are also available. Contact College Park Customer Service for more information.

CAST MODIFICATIONS/ SOCKET DESIGN

To prolong the life of the liner, a Total Surface Bearing-style socket is required. Excessive modification or buildups on the model can cause damage to the liner and reduce its lifespan. (It is acceptable to create a subtle patella tendon bar and/or posterior shelf to prevent rotation in the socket.)

WARNING

- Never wear the liner over open sores or non-intact skin.
- Do not wear the liner for more than 20 hours daily.
- Do not apply lotions or other products before donning the liner.
- If you notice discomfort, poor circulation, and/or changes in skin color, discontinue use and notify your prosthetist immediately.

For further questions, please contact College Park Technical Service.

WARRANTY INSPECTION / MAINTENANCE INFORMATION

College Park recommends that you schedule your patients for check-ups according to the warranty Inspection schedule below.

We recommend you visually inspect the entire prosthesis for excessive wear and fatigue at each consultation.

WARRANTY INSPECTION SCHEDULE FOR GENESEE GEL LINER: 6 MONTHS.

TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 am – 5:30 pm (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.

LIABILITY

The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer

CAUTION

EN

College Park products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards are achieved only when College Park products are used with other recommended College Park components. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients.

CAUTION

If any problems occur with the use of this product, immediately contact your medical professional. The prosthetist and/or patient should report any serious incident* that has occurred in relation to the device to College Park Industries, Inc. and the competent authority of the Member State in which the prosthetist and/or patient is established.

*“Serious incident” is defined as any incident that directly or indirectly led, may have led, or might lead to any of the following: (a) the death of a patient, user, or other person, (b) the temporary or permanent serious deterioration of a patient’s, user’s, or other person’s state of health, (c) a serious public health threat.

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©2022 College Park Industries, Inc. All rights reserved.