GENESEE GEL ROLL-ON

user guide


دليل المستخدم • brugervejledning • Handbuch • οδηγός χρήστη
manual del usuario • käyttöopas • Guide de l'utilisateur
מדרך למשתמש • guida per l'utente • gebruikershandleiding
guia do utilizador • guia do usuário • ρυководство пользователя
priručka pre použivatelů • användarguide • kullanim kılavuzu • 用户指南

technical instructions

التعليمات الفنية • Teknisk vejledning • Technische Anleitung • Τεχνικές οδηγίες • Instrucciones técnicas
Tekniset ohjeet • Instructions techniques • הוראות טכניות • Istruzioni tecniche • Technische instructies
Tekniske instruksjoner • Instrukcje techniczne • Instruções técnicas • Instruções técnicas • Технические
инструкции • Technické pokyny • Tekniska anvisningar • Teknik Talimatlar • 技术说明

FIGURE 1

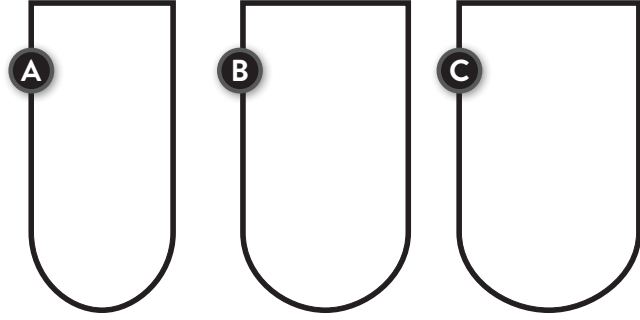Key Components (Figure 1)

(2) GENESEE GEL ROLL-ONS / SIZES:

A. Narrow

B. Medium

C. Wide

This diagram (Figure 1) is to help familiarize you with the unique parts of the Genesee Gel Roll-On. These parts are referenced in the instructions and used when speaking with a technical service representative.

PRODUCT DESCRIPTION

The Genesee gel roll-on is a thermoplastic elastomer (TPE) gel roll-on. Two roll-ons are included in each package.

INTENDED USE

Gel Roll-on is designed to assist with fit issues due to unique liner fitting or for additional padding.

INDICATIONS:

- Lower limb amputations

CONTRAINDICATIONS:

- None known

TECHNICAL SPECIFICATIONS

Material	Thermoplastic Elastomer (TPE) Gel
Sizes	Narrow, Medium, Wide
Thickness	2 mm
Overall Length	8.5 in

SIZE	NARROW	MEDIUM	WIDE
Circumference	18.5 cm (7.25 in)	21.5 cm (8.5 in)	23.5 cm (9.25 in)

The College Park's Genesee Gel Distal Roll-On is suitable to be used alone, or with other gel and fabric-faced prosthetic interfaces. The roll-on is made with hypoallergenic material with no additives that could cause skin reactions. It's also infused with a mineral oil gel to help against shear and friction. The roll-on will self-adhere to other Genesee Gel products (liners, suspension sleeves, etc).

FITTING GUIDELINES

Apply the distal roll-on alone, or underneath a gel liner in order to enhance socket fit. The roll-on can be cut to shape by the Prosthetist using sharp scissors or equivalent. Cut into a band of any width for added comfort under the proximal end of a suspension sleeve.

CLEANING GUIDELINES

Wash with soap and water before use.

Hand wash the roll-on using a mild detergent. Do not scrub the gel material. Rinse thoroughly and blot dry using a lint free cloth. Allow to dry completely before reuse.

WARNING

- Never wear the roll-on over open sores or non-intact skin.
- Do not wear the roll-on for more than 20 hours daily.
- Do not apply lotions or other products before donning the roll-on.
- If you notice discomfort, poor circulation, and/or changes in skin color, discontinue use and notify your prosthetist immediately.

For further questions, please contact College Park Technical Service.

INSPECTION INFORMATION

College Park recommends that you visually inspect the entire prosthesis for excessive wear and fatigue at each consultation.

WARRANTY INSPECTION SCHEDULE FOR GENESEE GEL ROLL-ON: NONE.

TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 am – 5:30 pm (EST).

After hours, an emergency Technical Service number is available to contact a College Park representative.

LIABILITY

The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

TERMS AND CONDITIONS

DAMAGES/LIABILITIES.

College Park is not liable for indirect, incidental, or consequential damages in connection with the use of the Genesee Gel Distal Roll-On. These damages include, without limitation, loss, damage, or expense attributable to the loss of use of the Genesee Gel Distal Roll-On, loss or damage to property other than the Genesee Gel Distal Roll-On, loss of expected income, loss from business disruption, or other commercial or personal loss due to any cause, even if College Park has been advised of the possibility of such damages, in connection with the Genesee Gel Distal Roll-On, or any use of the Genesee Gel Distal Roll-On (except for liability for consequential damages to the extent that by law such may not be disclaimed).

DISCLAIMER:

TO THE EXTENT PERMITTED BY LAW, COLLEGE PARK DISCLAIMS ALL WARRANTIES, EXPRESS AND IMPLIED, INCLUDING, WITHOUT LIMITATION, STATE LAW IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THERE ARE NO EXPRESS OR IMPLIED WARRANTIES, GUARANTEES, AGREEMENTS OR SIMILAR OBLIGATIONS OF COLLEGE PARK. REPRESENTATIONS, ORAL OR WRITTEN, ARE NOT AUTHORIZED AND, IF GIVEN, SHOULD NOT AND CANNOT BE RELIED UPON.

CAUTION

College Park products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards are achieved only when College Park products are used with other recommended College Park components. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients.

CAUTION

If any problems occur with the use of this product, immediately contact your medical professional. The prosthetist and/or patient should report any serious incident* that has occurred in relation to the device to College Park Industries, Inc. and the competent authority of the Member State in which the prosthetist and/or patient is established.

*“Serious incident” is defined as any incident that directly or indirectly led, may have led, or might lead to any of the following: (a) the death of a patient, user, or other person, (b) the temporary or permanent serious deterioration of a patient’s, user’s, or other person’s state of health, (c) a serious public health threat.

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©2022 College Park Industries, Inc. All rights reserved.