INSTRUCTION SHEET | ENDOSKELETAL COMPONENTS

Torsion Adapter

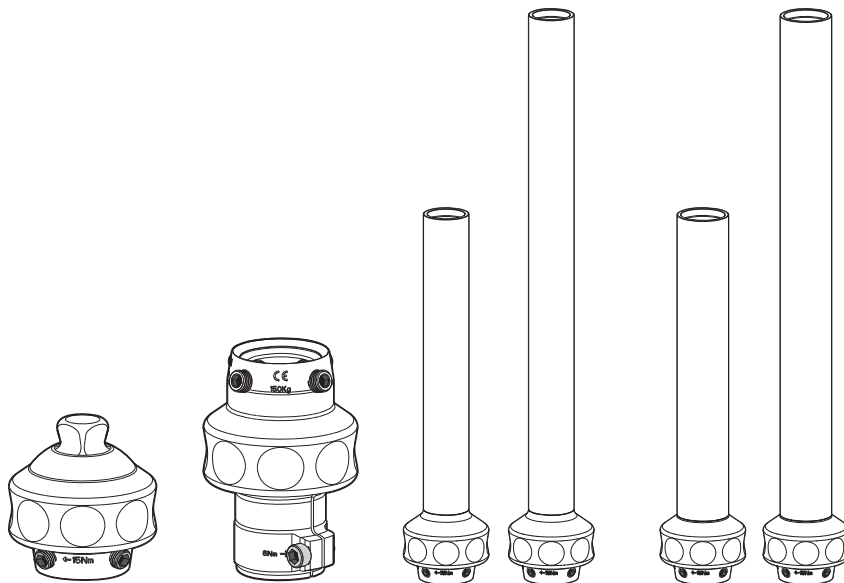
FIGURE 1


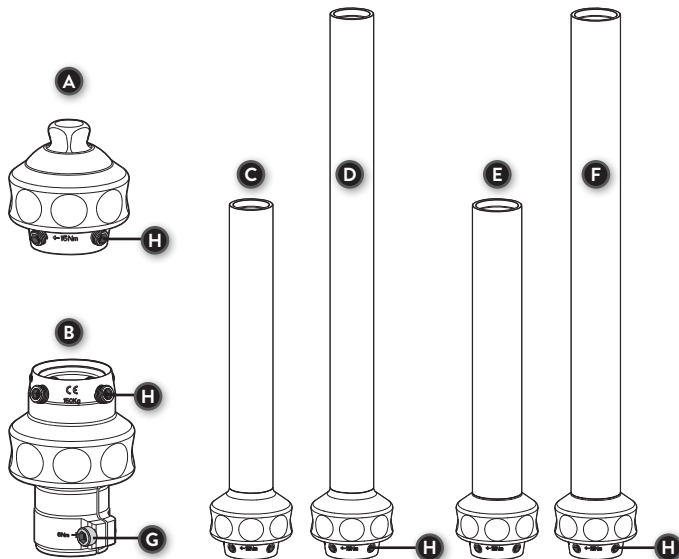
FIGURE 2

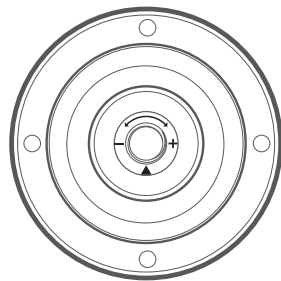INSTRUCTION SHEET | ENDOSKELETAL COMPONENTS

Torsion Adapter

PRODUCT DESCRIPTION

This structural component consists of a torsional mechanism with a pyramid receiver and four set screws on one connecting end. The other connecting end consists of one of the following: pyramid, pylon (tube), or pylon receiver with clamp screw.

KEY COMPONENTS (Figure 1)

Torsion Adapters

- A. Torsion Adapter, Pyramid Adapter to Pyramid Receiver, Ti
- B. Torsion Adapter, 30mm Tube Clamp to Pyramid Receiver, Ti
- C. Torsion Adapter, 30mm Tube Adapter, Ti,, 275mm
- D. Torsion Adapter, 30mm Tube Adapter, Ti, 455mm
- E. Torsion Adapter, 34mm Tube Adapter, Ti, 275mm
- F. Torsion Adapter, 34mm Tube Adapter, Ti, 455mm
- G. (1) M5 x 14mm Clamp Screw
- H. (4) M8 x 14mm Set Screws

This diagram (Figure 1) is to help familiarize you with a torsion adapter, a structural component. These parts are referenced in the instructions and used when speaking with a technical service representative.

INTENDED USE

Torsion Adapters are prosthetic adapters intended to mimic the torsional compliance of an anatomical limb by allowing for torsional rotation between the prosthetic socket and prosthetic foot.

INDICATIONS:

- Lower limb amputations

CONTRAINDICATIONS:

- None known

WARNING

- Torque evenly to specified settings.
- Do not expose this product to corrosive materials, salt water or pH extremes.
- Any further disassembly or modification of components will void the warranty.

Failure to follow these guidelines may result in one or all of the following: noise, movement, damage, and catastrophic failure. For further questions, please contact College Park Technical Service..

WARRANTY INSPECTION AND MAINTENANCE INFORMATION

College Park recommends that you schedule your patients for check-ups according to the Warranty Inspection schedule below.

High patient weight and/or impact level may require more frequent inspections. Prosthetic components should be inspected after the first 30 days of use. We recommend you visually inspect the entire prosthesis for excessive wear and fatigue at each consultation..

WARRANTY INSPECTION SCHEDULE FOR TORSION ADAPTER: 30 DAYS, THEN ANNUALLY.

TECHNICAL SPECIFICATIONS

	Torsion Adapter, Pyramid Adapter to Pyramid Receiver, Ti	Torsion Adapter, 30mm Tube Clamp to Pyramid Receiver, Ti	Torsion Adapter, 30mm Tube Adapter, Ti, 275mm	Torsion Adapter, 30mm Tube Adapter, Ti, 455mm	Torsion Adapter, 34mm Tube Adapter, Ti, 275mm	Torsion Adapter, 34mm Tube Adapter, Ti, 455mm
Part No.	TOR PAPR T	TOR TCPR T	TOR T275 30	TOR T455 30	TOR T275 34	TOR T455 34
Material	Titanium	Titanium	Titanium (receiver), Aluminum (tube)	Titanium (receiver), Aluminum (tube)	Titanium (receiver), Aluminum (tube)	Titanium (receiver), Aluminum (tube)
Tube Diameter	n/a	30 mm	30 mm	30 mm	34 mm	34 mm
Build Height (MAX)	43 mm (1.7 in)	47 mm (1.9 in)	275 mm (10.8 in)	455 mm (18 in)	275 mm (10.8 in)	455 mm (18 in)
Cut Length (MIN)	n/a	n/a	83 mm (3.3 in)	83 mm (3.3 in)	79 mm (3.1 in)	79 mm (3.1 in)
Assembly Weight	393 g	432 g	567 g	670 g	569 g	685 g
Patient Weight Limit	150 kg (330 lbs)	136 kg (300 lbs)	136 kg (300 lbs)	136 kg (300 lbs)	150 kg (330 lbs)	150 kg (330 lbs)
Axial Rotation	+/- 20°	+/- 20°	+/- 20°	+/- 20°	+/- 20°	+/- 20°
Warranty	2 Year, limited	2 Year, limited	2 Year, limited	2 Year, limited	2 Year, limited	2 Year, limited
Torque (Set Screws)	15 N·m (11ft·lbs)	15 N·m (11ft·lbs)	15 N·m (11ft·lbs)	15 N·m (11ft·lbs)	15 N·m (11ft·lbs)	15 N·m (11ft·lbs)
Torque (Clamp Screw)	n/a	6 N·m (4.4 ft·lbs)	n/a	n/a	n/a	n/a

TORSION ADJUSTMENTS *(Figure 2)*

- Range of adjustment = 2 full turns
- Factory setting = minimum resistance

SYMPTOM	DESIRED RESULT	ADJUSTMENT	
Rotation too fast	Less torsion moment/ Increase Resistance	Turn clockwise	
Rotation too slow	Greater torsion moment/ Decrease Resistance	Turn counter-clockwise	

GUIDELINES

M8 x 14mm set screw and M5 x 14mm clamp screw threads include nylon patch. Loctite® is not required.

TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 am – 5:30 pm (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.

LIABILITY

The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer

CAUTION

College Park products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards are achieved only when College Park products are used with other recommended College Park components. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients.

CAUTION

If any problems occur with the use of this product, immediately contact your medical professional. The prosthetist and/or patient should report any serious incident* that has occurred in relation to the device to College Park Industries, Inc. and the competent authority of the Member State in which the prosthetist and/or patient is established.

*‘Serious incident’ is defined as any incident that directly or indirectly led, may have led, or might lead to any of the following; (a) the death of a patient, user, or other person, (b) the temporary or permanent serious deterioration of a patient's, user's, or other person's state of health, (c) a serious public health threat.

COMPLIANCE

This device has been tested according to ISO 10328 standard to three million load cycles.

Depending on patient activity this may correspond to 3-5 years of use.

ISO 10328 - LABEL

PART NUMBER	WEIGHT LIMIT (KG)	LABEL TEXT
TOR TCPRT		
TOR T275 30	136	ISO 10328-P6-136 kg
TOR T455 30		
TOR PAPRT		
TOR T275 34	150	ISO 10328-P7-150 kg
TOR T455 34		

ISO 10328 - “P” - “m”kg*)

**) Body mass limit not to be exceeded!*

For specific conditions and limitations of use, see intended use section of manufacturer's written instructions.

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©2023 College Park Industries, Inc. All rights reserved.