


COSMETIC FINISHING

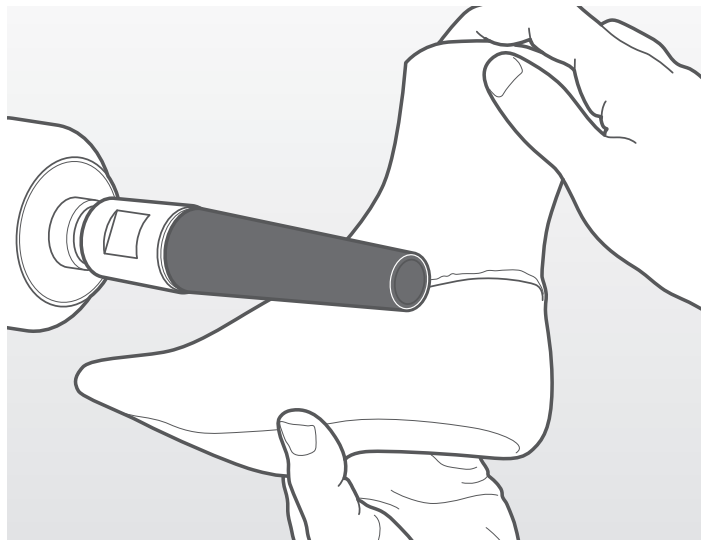

Lightly grind the ankle fairing at the transition point with the foot shell to smooth the lip. Then cosmetically finish the leg using your preferred finishing method.



COLLEGE PARK INDUSTRIES
27955 College Park Dr. Warren,
MI 48088 USA


EMERGO EUROPE
Prinsessegracht 20, 2514 AP
The Hague, Netherlands

Australian Sponsor
EMERGO AUSTRALIA
Level 20, Tower II
Darling Park
201 Sussex Street
Sydney, NSW 2000
Australia




©2017 College Park Industries, Inc. All rights reserved. College Park is
a registered trademark of College Park Industries, Inc.

INSTRUCTION SHEET

Accent® Ankle Fairing / Cosmetic Finishing

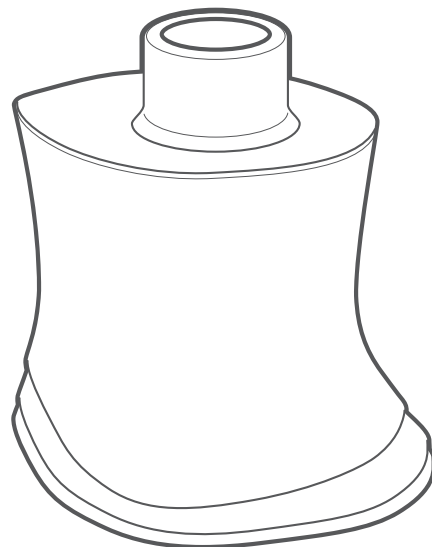

Required for both Dynamic Pylon (DP) and Integrated Pyramid (IP) options, to provide a smooth transition for cosmetic finishing and internal protection from environmental elements.

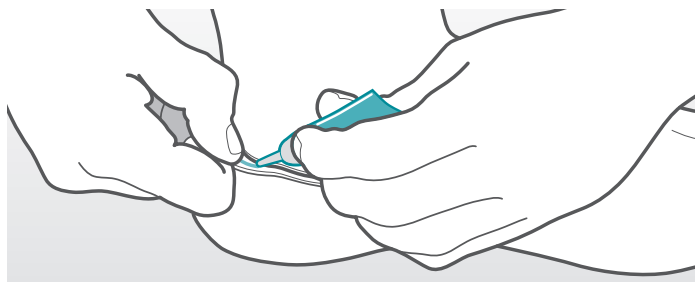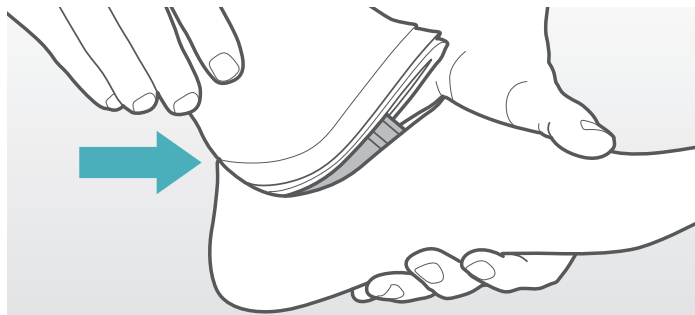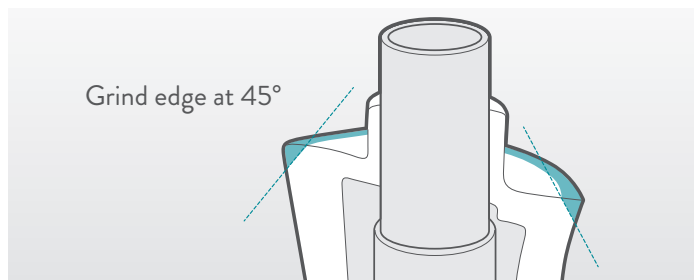

ANKLE FAIRING INSTALLATION

- 1 You may grind the top of the ankle fairing and the distal foam to match (slightly undersized) at a 45° angle to improve shaping at the transition point.
- 2 To facilitate insertion of the ankle fairing into the foot shell, begin at the heel and work toward the toes. Use caution when placing the ankle fairing over endoskeletal components to avoid damage to the ankle fairing.
- 3 Pull back the “lip” of the foot shell and apply a continuous bead of the supplied Cyanoacrylate (CA) adhesive around the inside lip of the foot shell. It is important to get a 360 degree bond.
- 4 Lightly rotate the ankle fairing to ensure proper seating and bonding, pressing on the area around the bond to adhere.



CAUTION

- Ankle fairing must be used to prevent dirt and debris from entering the ankle mechanism. Failure to use the fairing may void the warranty.
- It is recommended to use the supplied cyanoacrylate (CA) adhesive.
- Permanent bonding of the ankle fairing is required to prevent misalignment.