

INSTRUCTION SHEET

Cosmetic Attachment Plate (CAP)

TOOLS NEEDED

- Cosmetic Attachment Plate (CAP)
- College Park Enviroshell
- Master All-Purpose Cement (or any all-purpose waterproof adhesive)
- Sandpaper

INSTRUCTIONS

- 1 Sand the top surface of the CAP to create a rough contact surface for proper adhesion.
- 2 Use contact adhesive to bond the cosmetic cover to the CAP.
- 3 hape soft cosmetic cover as preferred
- 4 Bond it to the foot shell with four evenly spaced drops of Cyanoacrylate (CA) adhesive. Hold CAP in place until the glue is dry.
- 5 Finish with choice of prosthetic skin or nylon.

For further questions, please contact College Park Technical Service.



college park

800.728.7950 | 586.294.7950 | college-park.com



COLLEGE PARK INDUSTRIES

27955 College Park Dr. Warren, MI 48088 USA



EMERGO EUROPE

Prinsessegracht 20, 2514 AP The Hague, Netherlands

Australian Sponsor EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex Street
Sydney, NSW 2000 Australia

MADE IN THE USA

©2017 College Park Industries, Inc. All rights reserved. College Park is a registered trademark of College Park Industries, Inc.



595 INS CAP 170804