

INSTRUCTION SHEET

FINISHING RING

INSTRUCTIONS

- 1 Remove the ankle bone from the exo block.
- 2 Install up to two growth plates, making sure the anti-rotation pins are positioned correctly. The pins must be positioned alternately in the anterior or posterior location depending on the number of growth plates used.

The growth plates must assemble flush, with no interference, for correct foot application.

- 3 Use the finishing ring to close the gap presented upon leg lengthening. Re-assemble the ankle bone to the growth plates and exo block.
- 4 Apply removable Loctite 242 to mounting bolt.
Torque Mounting Bolt 61 N-m (45 ft-lb)

For further questions, please contact College Park Technical Service.



college park

800.728.7950 | 586.294.7950 | college-park.com

MADE IN THE USA



COLLEGE PARK INDUSTRIES

27955 College Park Dr. Warren, MI 48088 USA



EMERGO EUROPE

Prinsessegracht 20, 2514 AP The Hague, Netherlands

Australian Sponsor EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex Street
Sydney, NSW 2000 Australia



©2017 College Park Industries, Inc. All rights reserved. College Park is a registered trademark of College Park Industries, Inc.

594 INS FR 170804