
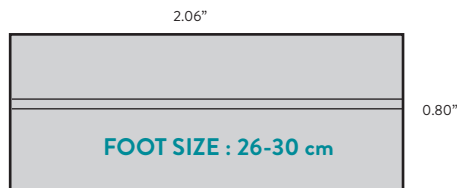



## SIZING OPTIONS FOR HEEL WEDGE

(Shown at actual size)

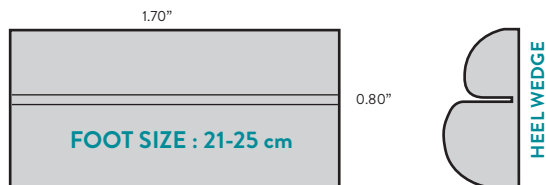

## HORIZON® HD WEDGE KIT

For use with Horizon HD, Odyssey® iQ, Odyssey® K3, Orbit®, and Terrain® feet

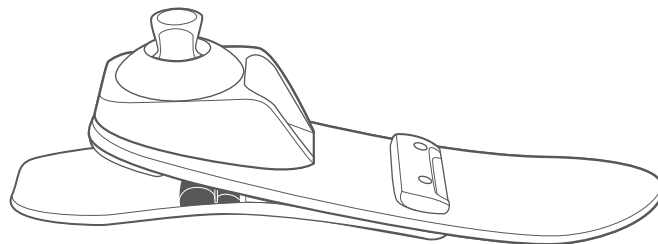

800.728.7950 | 586.294.7950 | college-park.com



416 INS HZ WK 250124

COLLEGE PARK INDUSTRIES, INC  
27955 College Park Dr. Warren, MI 48088 USA

EMERGO EUROPE

Westervoortsedijk 60, 6827 AT Arnhem, The Netherlands



MEDENVOY SWITZERLAND

Gotthardstrasse 28, 6302 Zug, Switzerland



MADE IN THE USA

©2025 College Park Industries, Inc. All rights reserved.

Australian Sponsor  
EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex Street,  
Sydney, NSW 2000 Australia




college park

TECHNOLOGY for the HUMAN RACE

instruction  
sheet


## HORIZON WEDGE PLACEMENT

For use with Horizon HD, Odyssey iQ, Odyssey K3, Orbit, and Terrain feet

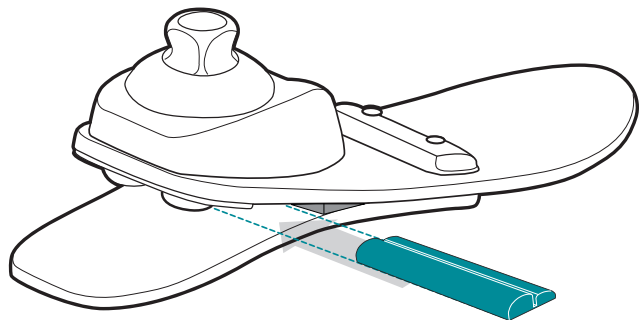

## ADJUSTMENT OPTIONS FOR WEDGES

HEEL FIRMNESS	WEDGE
MEDIUM	NONE*
FIRM	Single
X-FIRM	Double

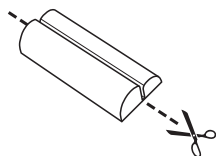

Make a Single Wedge by cutting the Double Wedge in half.

\*NOTE: Medium build comes standard. There is a triangular shaped wedge permanently bonded into all feet.

## WEDGE INSTALLATION AND REMOVAL

### SEMI-PERMANENT BOND (INITIAL FITTING)

- 1 Apply only the smallest amount of CA adhesive\* necessary to temporarily hold the wedge into position. Too much will inhibit removal.
- 2 Install the rounded wedge directly behind the triangular wedge.
- 3 Press on the wedge and hold it for 15 seconds.

### WEDGE REMOVAL

To remove the rounded wedge without damage, pry it away with a finger or use a flat, blunt object such as a tongue depressor.

### PERMANENT BOND (FINAL FITTING)

- 1 Apply a thin line of CA adhesive\* around the bottom perimeter of the wedge.
- 2 When installing the rounded wedge, place directly behind the triangular wedge.
- 3 Press on the wedge and hold it for 15 seconds.

**NOTE:** The wedge will NOT be removable once it has been bonded.



### CAUTIONS

- Use only the correct size wedge in the foot (see back for sizing guide).
- Do not remove permanent triangular wedge.
- Place wedge only in the recommended position.
- Do not substitute other objects in place of recommended wedges.

\*Cyanoacrylate (CA) adhesive, included as part of the wedge kit.

