



Partial Hand SOLUTIONS, LLC



Technical Instructions **Pediatric M-Finger** *Available in 3, 4 or 5 finger kits only*

PACKAGE CONTENTS

M-Finger Assembly
(3, 4, or 5 finger kits)
Spectra Cable (installed)
1/16" Allen wrench
Mounting hardware packet

Spare Parts
(1) BOA Cable Connector
(1) DCM - Direct
Connection Mechanism
(1) Gaffney joint

RECOMMENDED TOOLS AND SUPPLIES

Tweezers
Phillips screwdriver
Superglue
Razor blade to cut cable
Trautman - grinding machine



3 Finger Kit

1. Mounting bracket
2. Gaffney
3. Finger mounting screws and nuts (x2)
4. DCM-3
5. BOA



4 Finger Kit

1. Mounting bracket
2. Gaffney
3. Finger mounting screws and nuts (x3)
4. DCM-4
5. BOA



5 Finger Kit

1. Mounting bracket
2. Gaffney
3. Finger mounting screws and nuts (x3)
4. DCM-4
5. BOA

Mounting Hardware

Finger mounting screws: 4-40 x 1/2"

Mounting plate screws: Phillips flat head screw, 4-40 x 1/2"

Spare Parts:

(2) Flat head solid rivets: 3/32" x .328

(2) Retaining rings: 3/32"

(1) Cable set screw: 6-32 x 3/16"

(2) Cable rollers: 1/8" x 1/8"

(2) Roller screws: 0-80 x 3/8"

(2) Pediatric M-Finger springs: 1/2"

Mounting Instructions

1. Use supplied screws and insert screws through holes in mounting bracket and through the lamination (hole in lamination should be 5/32 to allow for male end of nut to recess in to lamination)
2. Apply loctite 242 to nuts and hand tighten.

M-Fingers are distributed by:
 **college park**
800-728-7950 | 586-294-7950
27955 College Park Drive, Warren, MI 48088

TECHNICAL ASSISTANCE/ EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 am – 5:30 pm (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.



Partial Hand
SOLUTIONS, LLC



Partial Hand Solutions
86 Ladyslipper Lane
Southington, CT USA 06489
+1 (860) 538-5532



EMERGO EUROPE
Prinsessegracht 20,
2514 AP
The Hague, Netherlands

MADE IN THE USA
895 INS PHS MFP 190815

