
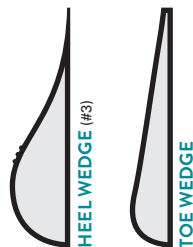
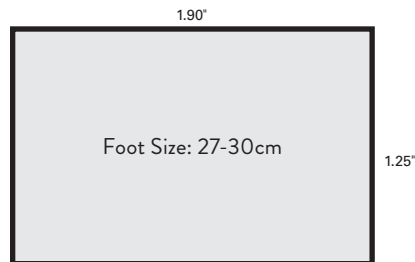


## SIZING OPTIONS FOR HEEL AND TOE WEDGES

(Shown at actual size)

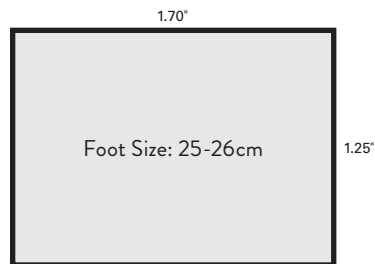

**EMERGO EUROPE**  
Prinsessegracht 20, 2514 AP  
The Hague, Netherlands

**MADE IN THE USA**  
109 INS VL WK 170823




**Australian Sponsor**  
**EMERGO AUSTRALIA**  
Level 20, Tower II, Darling Park  
201 Sussex Street  
Sydney, NSW 2000 Australia

©2017 College Park Industries, Inc. All rights reserved. College Park is a registered trademark of College Park Industries, Inc.


# INSTRUCTION SHEET

## VELOCITY WEDGE KIT

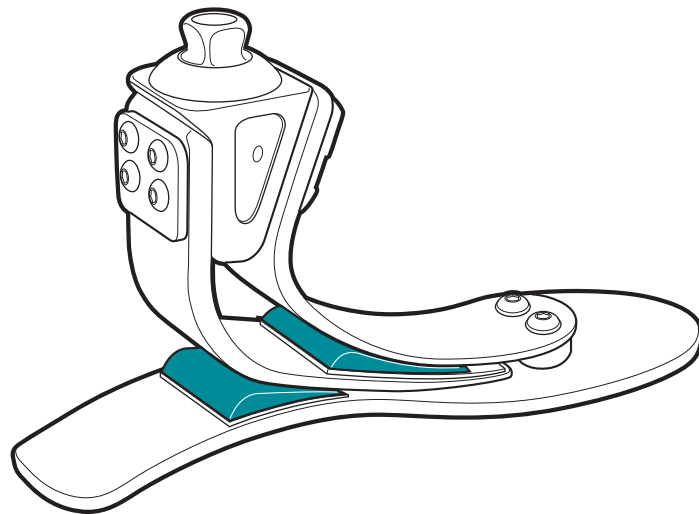


Heel and toe wedges are not required.  
Follow these guidelines if a change in firmness is desired.

## VELOCITY WEDGE PLACEMENT

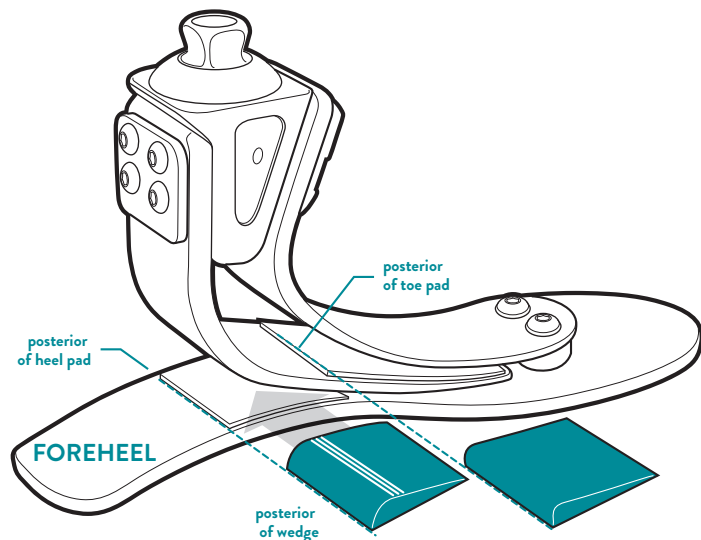

## ADJUSTMENT OPTIONS FOR WEDGES

HEEL FIRMNESS	WEDGE #	INDICATORS
STANDARD	NONE	NONE
MEDIUM	1	
FIRM	2	
X-FIRM	3	
TOE FIRMNESS		
FIRMER	ADD TOE WEDGE	

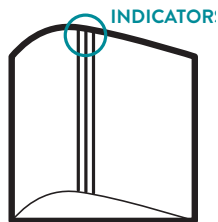

## WEDGE INSTALLATION AND REMOVAL

### SEMI-PERMANENT BOND (INITIAL FITTING)

- 1 Apply only the smallest amount of CA adhesive\* necessary to temporarily hold the wedge into position. Too much will inhibit removal.
- 2 When installing the wedge, align posterior edges of the wedge and pad.
- 3 Press on the wedge and hold it for 15 seconds.




### WEDGE REMOVAL

To remove the wedge without damage, pry it away with a finger or use a flat, blunt object such as a tongue depressor.

### PERMANENT BOND (FINAL FITTING)

- 1 Apply a thin line of CA adhesive\* around the bottom perimeter of the wedge.
- 2 When installing the wedge, align posterior edges of the wedge and pad.
- 3 Press on the wedge and hold it for 15 seconds.

**NOTE:** The wedge will NOT be removable once it has been bonded.

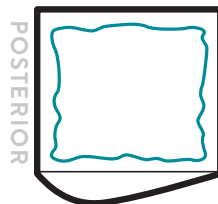

### ⚠ CAUTIONS

- Use only the correct size wedge in the foot (see back for sizing guide).
- Do not install more than one wedge into the heel or toe positions.
- Pre-installed toe wedge can not be removed.
- Place wedge only in the recommended position.
- Do not force the wedge further into the toe springs.
- Do not substitute other objects in place of recommended wedges.

\*Cyanoacrylate (CA) adhesive, included as part of the wedge kit.