

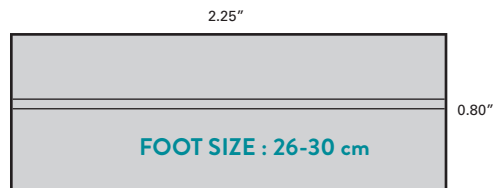


## SIZING OPTIONS FOR HEEL WEDGE

(Shown at actual size)

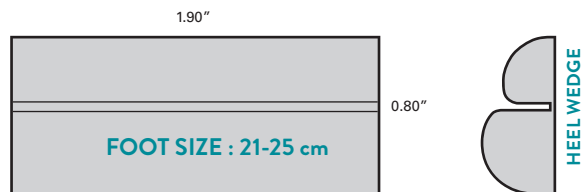

## ODYSSEY®<sub>K3</sub> WEDGE KIT

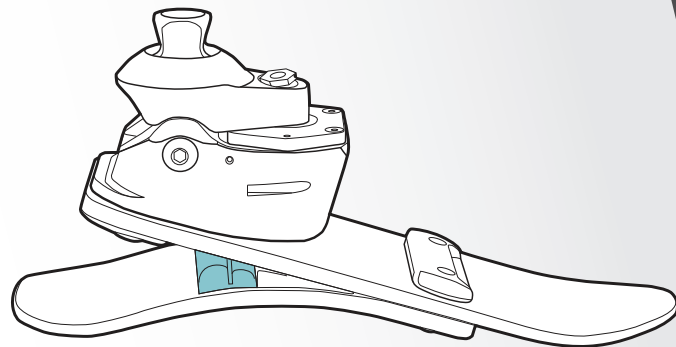

800.728.7950 | 586.294.7950 | college-park.com

**COLLEGE PARK INDUSTRIES, INC**  
27955 College Park Dr., Warren, MI 48088 USA

**EMERGO EUROPE**  
Prinsessegracht 20, 2514 AP, The Hague, Netherlands



MADE IN THE USA

521 INS OK3 WK 180109

**Australian Sponsor: EMERGO AUSTRALIA**  
Level 20, Tower II, Darling Park  
201 Sussex Street  
Sydney, NSW 2000 Australia



college park


TECHNOLOGY for the HUMAN RACE

instruction  
sheet

©2018 College Park Industries, Inc. All rights reserved. College Park is a registered trademark of College Park Industries, Inc.

## ODYSSEY<sub>K3</sub> WEDGE PLACEMENT

Heel wedges are not required. Follow these guidelines if an increase in firmness is desired.

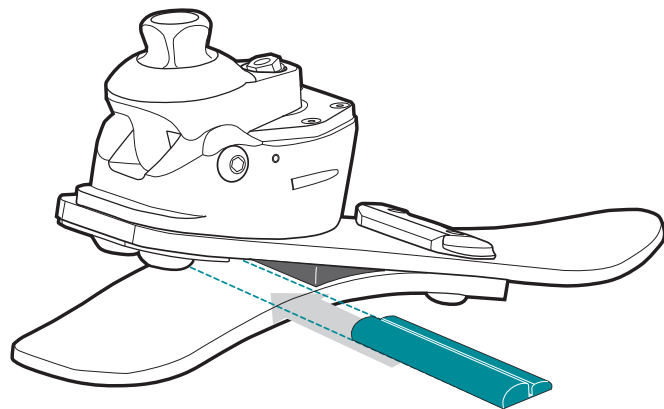

## ADJUSTMENT OPTIONS FOR WEDGES

### HEEL FIRMNESS

MEDIUM

FIRM

X-FIRM


### WEDGE

NONE\*

Single

Double

Make a Single Wedge by cutting the Double Wedge in half.

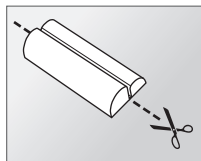


\*NOTE: Medium build comes standard. There is a triangular shaped wedge permanently bonded into all feet.

## WEDGE INSTALLATION AND REMOVAL

### SEMI-PERMANENT BOND (INITIAL FITTING)

- 1 Apply only the smallest amount of CA adhesive\* (included) necessary to temporarily hold the wedge into position. Too much will inhibit removal.
- 2 Install the rounded wedge directly behind the triangular wedge.
- 3 Press on the wedge and hold it for 15 seconds.

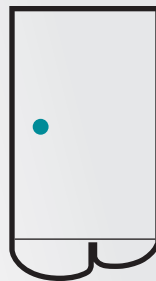


### WEDGE REMOVAL

To remove the rounded wedge without damage, pry it away with a finger or use a flat, blunt object such as a tongue depressor.

### PERMANENT BOND (FINAL FITTING)

- 1 Apply a thin line of CA adhesive\* around the bottom perimeter of the wedge.
- 2 When installing the rounded wedge, place directly behind the triangular wedge.
- 3 Press on the wedge and hold it for 15 seconds.

NOTE: The wedge will NOT be removable once it has been bonded.

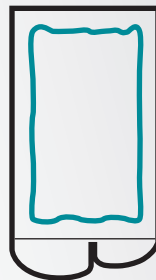

### CAUTIONS

- Use only the correct size wedge in the foot (see back for sizing guide).
- Do not remove permanent triangular wedge.
- Place wedge only in the recommended position.
- Do not substitute other objects in place of recommended wedges.

\*Cyanocrylate (CA) adhesive, included as part of the wedge kit.