get the rhythm back

volte ao ritmo • Kommen Sie wieder in Ihren Rhythmus • recuperare el ritmo
استعيد انزائك من جديد • reprenez le rythme • recupere o ritmo
восстановите ритм • 找回节奏

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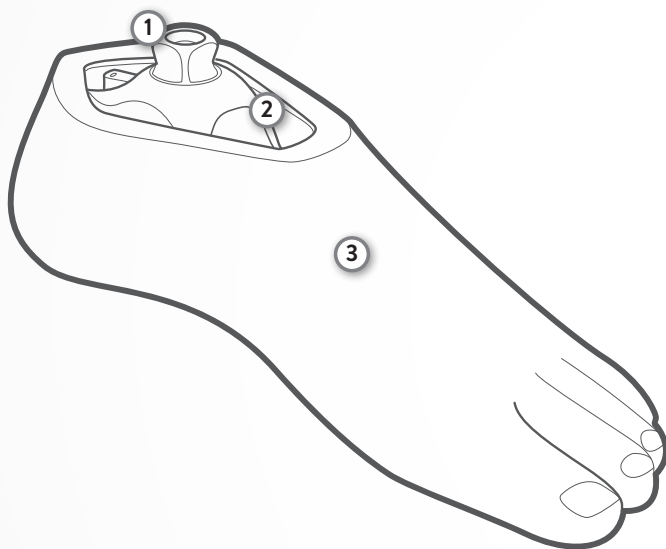
This diagram is to help familiarize you with the unique parts of the Trés. These parts are referenced in the instructions and used when speaking with a technical service representative.

KEY COMPONENTS

1. Integrated Pyramid
2. Housing (Ankle)
3. Bonded Foot Shell

PACKAGE CONTENTS

(1) Trés foot, with bonded foot shell

TOOLS RECOMMENDED

4mm Hex Key

WARNING

- Do not expose this product to pH extremes or corrosive materials (water, salt water or other liquids).
- Failure to follow these technical instructions or use of this product outside the scope of its Limited Warranty may result in injury to the patient or damage to the product.
- Disassembly, modification or removal from the foot shell, or any further disassembly or modification of components will void the warranty.

PROTECTIVE COVER ON DOME

Remove the protective cover on dome after alignment is completed and before patient leaves clinic.

MOUNTING

Use only high quality proximal endoskeletal components (30 mm). The service-free Trés foot comes permanently assembled in the foot shell.

STATIC ALIGNMENT

For optimal function have the patient's weight balanced between the forefoot and heel. The load line divides the foot at the 1/3 heel lever and 2/3 toe lever.

DYNAMIC ADJUSTMENTS

DESIRED RESULT	ALIGNMENT CHANGE
Firmer toe response	Plantarflex the Trés or move load line posterior
Softer toe response	Dorsiflex the Trés or move load line anterior
Firmer heel response	Dorsiflex the Trés or move load line anterior
Softer heel response	Plantarflex the Trés or move load line posterior

GAIT MATCHING® GUIDELINES

Contact College Park Technical Service if you have questions about modifying the gait match.

WARRANTY INSPECTION / MAINTENANCE INFORMATION

College Park recommends that you schedule your patients for check-ups according to the warranty inspection schedule below.

High patient weight and/or impact level may require more frequent inspections. We recommend you visually inspect the following applicable parts for excessive wear and fatigue at each warranty inspection.

- Composite and Adapter
- Foot Shell

WARRANTY INSPECTION SCHEDULE FOR TRÉS : ANNUALLY.

TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

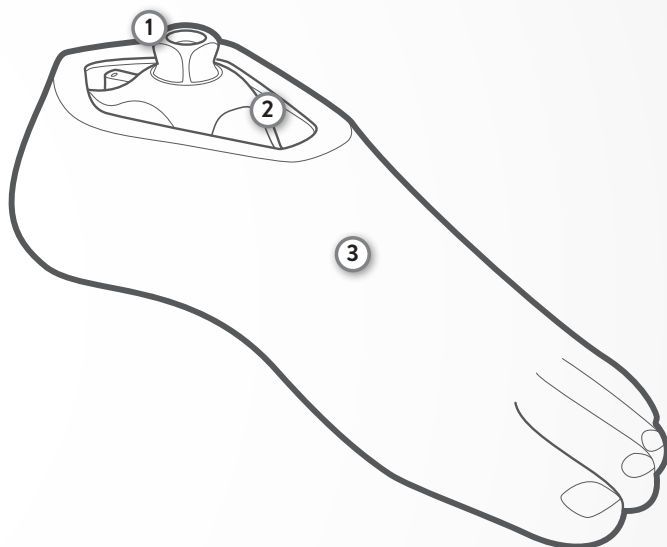
College Park's regular office hours are Monday through Friday, 8:30 am – 5:30 pm (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.

LIABILITY

The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

CAUTION

College Park products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when College Park products are used with other recommended College Park components. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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©2018 College Park Industries, Inc. All rights reserved. College Park, Tres, and Gait Matching are registered trademarks of College Park Industries.