



## TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 AM – 5:30 PM (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.



**COLLEGE PARK  
INDUSTRIES, INC.**  
27955 College Park Dr.  
Warren, MI 48088 USA



**EMERGO EUROPE**  
Prinsessegracht 20, 2514 AP  
The Hague, Netherlands

**Australian Sponsor  
EMERGO AUSTRALIA**  
Level 20, Tower II  
Darling Park  
201 Sussex Street  
Sydney, NSW 2000  
Australia

©2018 College Park Industries, Inc. All rights reserved. College Park and Accent are registered trademarks of College Park Industries. US Patent No. 7,862,622.



**MADE IN THE USA**  
055 INS ACSI 171109



accent<sup>®</sup> foot  
sizing guide

# accent® Foot Sizing Guide

		Width (cm)							
		21	22	23	24	25	26	27	28
ACCENT	Narrow								
	Wide								
U.S. Shoe Sizes	Youth/Men	2-2.5	3-4	4.5-5	5.5-7	7.5-8	8.5-9	9.5-10.5	11-11.5
	Women	4-4.5	5-5.5	6-6.5	7-8	8.5-9.5	10-10.5	11-11.5	12

## FOOTWEAR SELECTION



Please consider your patient's footwear preferences when determining foot shell width.

### WIDE FOOT SHELLS

Generally are preferred for loafers, boots, sneakers and orthopedic shoes. Lace-up shoes may have more flexibility, as you can control the width by how much the laces are tightened.

### NARROW FOOT SHELLS

Generally are preferred for high heeled shoes (pointy toes and high-fashion styles).

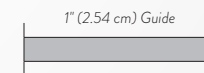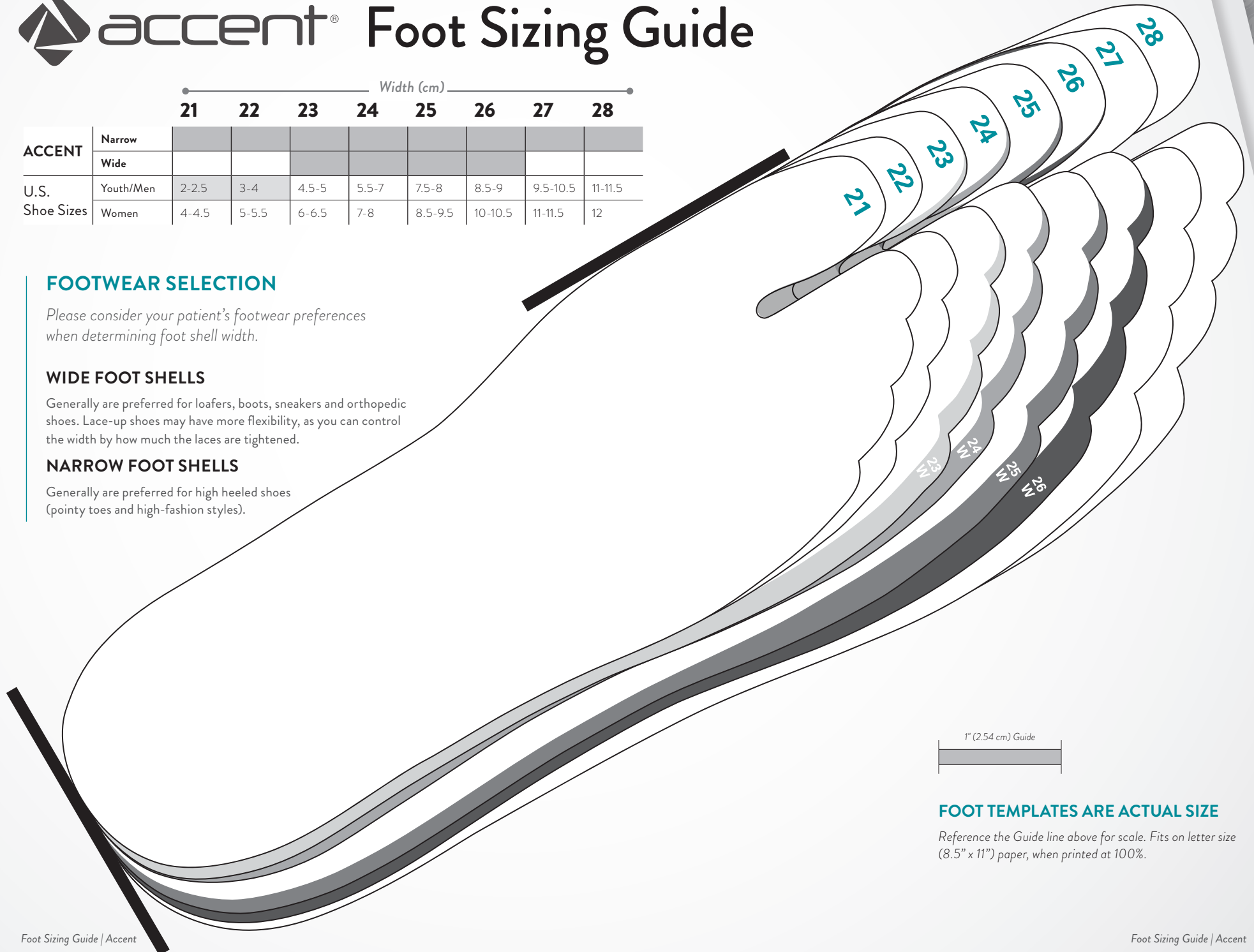

## FOOT TEMPLATES ARE ACTUAL SIZE

Reference the Guide line above for scale. Fits on letter size (8.5" x 11") paper, when printed at 100%.