college park
TECHNOLOGY *for the HUMAN RACE*

technical instructions
Eastern Europe

FIGURE 1


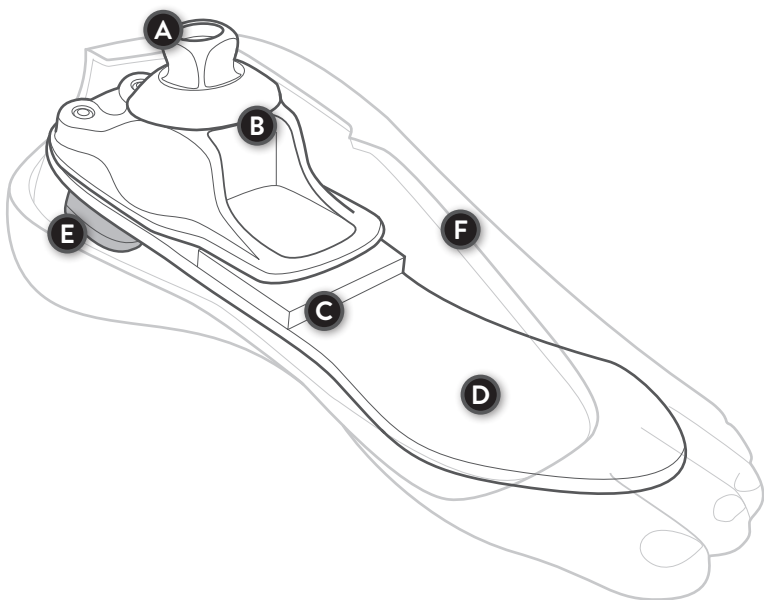
FIGURE 2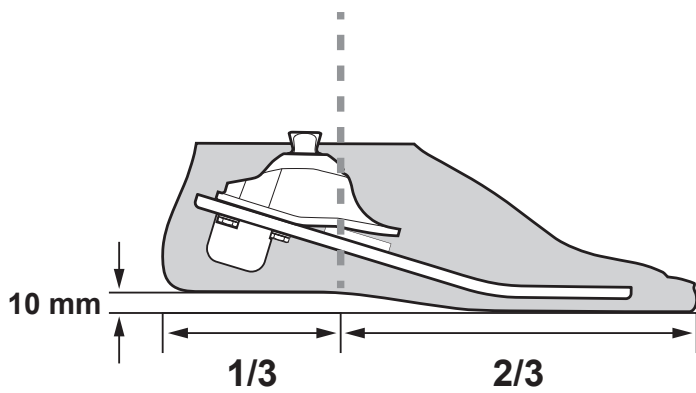

PACKAGE CONTENTS

(1) Trés foot, with bonded foot shell

TOOLS RECOMMENDED

4mm Hex Key

EN

This diagram is to help familiarize you with the unique parts of the Trés. These parts are referenced in the instructions and used when speaking with a technical service representative.

Key Components (Figure 1)

A. Integrated Pyramid

B. Housing

C. Fulcrum Pad

D. Toe Spring

E. Heel Bumper

F. Bonded Foot Shell

PRODUCT DESCRIPTION

This prosthetic foot device is constructed with an integrated pyramid, housing, composite toe spring, and heel bumper. The toe spring is secured to the housing with fasteners. The foot shell is bonded to the toe spring and is not removable.

INTENDED USE

The Trés is a prosthetic foot designed to replace one or more functions of the biologic human foot.

⚠ INDICATIONS:

- Lower limb amputations

⚠ CONTRAINDICATIONS:

- None known

⚠ PROTECTIVE COVER ON DOME

Remove the protective cover on dome after alignment is completed and before patient leaves clinic.

TECHNICAL SPECIFICATIONS

FOOT SIZE	WEIGHT LIMIT	BUILD HEIGHT	FOOT WEIGHT*
21-24 cm	300 lbs / 136 kg	2.5 in / 6.4 cm	500 g
25-30 cm	330 lbs / 150 kg		

*26cm foot w/shell

GAIT MATCHING® GUIDELINES

The gait match determines the firmness of the foot based on the user's specifications (foot size, patient weight, and impact level).

FIRMNESS CATEGORIES

Refer to the chart below to determine the correct firmness category.

Note: Incorrect category selection may result in poor device function. Contact College Park Technical Service if you have questions about category selection.

WEIGHT LBS	0-120	121-140	141-160	161-180	181-200	201-220	221-250	251-275	276-300	301-330
WEIGHT KG	0-54	55-63	64-72	73-81	82-91	92-100	101-113	114-125	126-136	137-150
SIZE CM	21-24									
Low Impact	1	1	2	2	3	3	4	4	4	N/A
Moderate Impact	1	2	2	3	3	3	4	4	4	N/A
SIZE CM	25-30									
Low Impact	1	1	2	2	3	3	4	4	4	5
Moderate Impact	1	2	2	3	3	3	4	4	5	5

MOUNTING

Use only high quality proximal endoskeletal components. The service-free Trés foot comes permanently assembled in the foot shell.

STATIC ALIGNMENT *(Figure 2)*

For optimal function, balance the patient's weight evenly between the heel and toe.

- The Trés was designed with a 3/8" (10 mm) heel rise.
- The load line divides the foot at 1/3

DYNAMIC ADJUSTMENTS

DESIRED RESULT	ALIGNMENT CHANGE
Firmer toe response	Plantarflex the Trés or move load line posterior
Softer toe response	Dorsiflex the Trés or move load line anterior
Firmer heel response	Dorsiflex the Trés or move load line anterior
Softer heel response	Plantarflex the Trés or move load line posterior

WARNING

- Do not expose this product to pH extremes or corrosive materials (water, salt water or other liquids).
- Failure to follow these technical instructions or use of this product outside the scope of its Limited Warranty may result in injury to the patient or damage to the product.
- Disassembly, modification or removal from the foot shell, or any further disassembly or modification of components will void the warranty.

WARRANTY INSPECTION / MAINTENANCE INFORMATION

College Park recommends that you schedule your patients for check-ups according to the warranty Inspection schedule below.

High patient weight and/or impact level may require more frequent inspections. We recommend you visually inspect the following applicable parts for excessive wear and fatigue at each warranty inspection.

- Composite and Adapter
- Foot Shell

WARRANTY INSPECTION SCHEDULE FOR TRÉS : ANNUALLY.

TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 am – 5:30 pm (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.

LIABILITY

The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

CAUTION

College Park products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards are achieved only when College Park products are used with other recommended College Park components. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients.

CAUTION

If any problems occur with the use of this product, immediately contact your medical professional. The prosthetist and/or patient should report any serious incident* that has occurred in relation to the device to College Park Industries, Inc. and the competent authority of the Member State in which the prosthetist and/or patient is established.

*'Serious incident' is defined as any incident that directly or indirectly led, may have led, or might lead to any of the following; (a) the death of a patient, user, or other person, (b) the temporary or permanent serious deterioration of a patient's, user's, or other person's state of health, (c) a serious public health threat.

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©2021 College Park Industries, Inc. All rights reserved.