


TROUBLESHOOTING TIPS

LINER ROLLING DOWN ON LIMB

May indicate a size or length problem. See your prosthetist to resolve.

SWEATING INSIDE OF LINER

This is common when first using a liner, or during strenuous activities. Remove the liner, dry it and your limb with a soft towel, and reapply the liner. Consult your prosthetist if the problem persists, or before trying antiperspirants, etc.

 **Warning:** If you ignore this problem, slippage of the liner may occur and could result in loss of prosthesis.

SKIN IRRITATION ALONG THE TOP EDGE OF THE LINER

This can occur if the liner is pulled onto the limb, instead of being rolled, or if there is a size issue. Ensure proper donning, and if this does not resolve the problem, contact your prosthetist.


SKIN IRRITATION INSIDE OF LINER

There may be multiple causes for this, including but not limited to:

- Failure to clean liner daily or rinse properly. Review the daily maintenance guidelines in this manual.
- Applying lotion or other products prior to donning. Do not apply lotions or any other products to skin prior to donning. If lotion or other products are needed/ desired, apply them at night, after doffing the liner.
- Bacterial or fungal infection of the skin or liner. Contact your prosthetist immediately. They may direct you to a dermatologist.

GENERAL DISCOMFORT IN LINER OR PROSTHESIS. EXCESSIVE REDNESS/SORES ON LIMB.

This is likely a fitting issue. Contact your prosthetist immediately.

 **WARNING:** Never wear the liner over open sores or non-intact skin.



800.728.7950 | 586.294.7950 | college-park.com

MADE IN THE USA

336 INS UG GN 170823



 **COLLEGE PARK INDUSTRIES, INC**
27955 College Park Dr.
Warren, MI 48088 USA


EC REP EMERGO EUROPE
Prinsessegracht 20, 2514 AP
The Hague, Netherlands

**Australian Sponsor
EMERGO AUSTRALIA**
Level 20, Tower II
Darling Park
201 Sussex Street
Sydney, NSW 2000
Australia



college park
metro liners
genesee

USER GUIDE

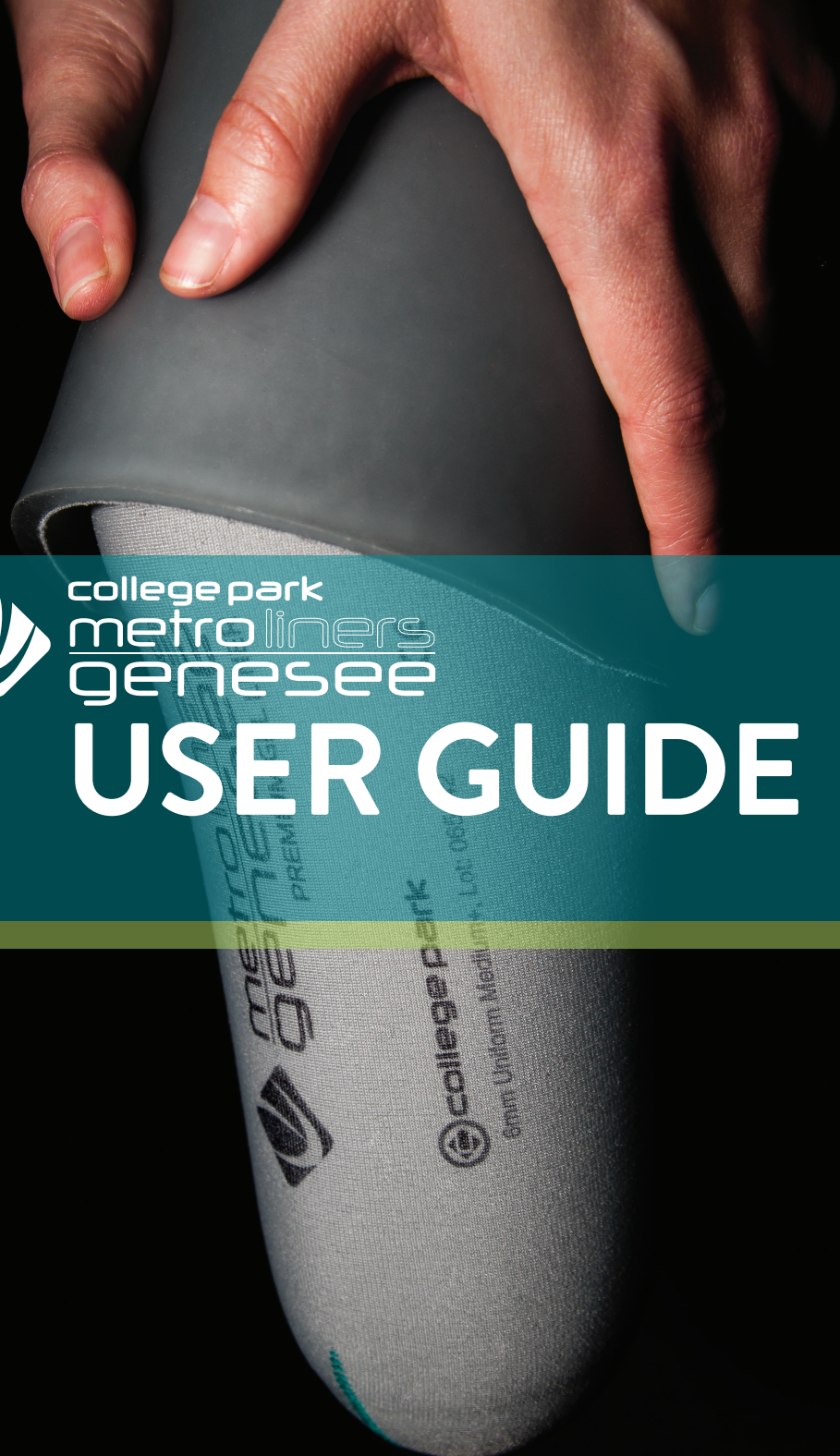

Thank you for using College Park's Genesee Gel liners.

The Genesee liner is made with...

- A mineral oil gel that protects against shear and friction, releasing mineral oil over the life of the liner to moisturize skin and scar tissue.
- Hypoallergenic material – does not contain additives that can cause skin reactions.
- A unique fabric that reduces pistoning, while allowing for circumferential stretch.
- Provides comfort and absorbs forces – so your skin won't have to.

INSTRUCTIONS FOR USE

LOCKING LINER INSTRUCTIONS

Each liner must have its own pin installed and properly secured by your prosthetist. Do not remove or attempt to tighten the pin. Inspect the pin daily and if it is found to be loose, do not use the liner. Contact your prosthetist immediately.

⚠ WARNING: Use of an improperly secured pin may lead to noise, separation, and potential injury.


DONNING THE LINER



Invert the liner completely (gel facing out). Align the bottom of the liner to the distal end of limb, with the logo facing front.



Roll liner gently onto limb. Do not pull on the liner (like a sock) as this may cause air pockets, skin irritation, wrinkles, etc.

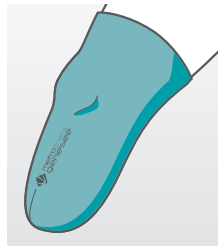

Ensure that air pockets have been eliminated and that there are no wrinkles in the liner.

MAINTENANCE

DAILY



Inspect the liner for signs of excessive wear to the gel and fabric, such as fraying, tears, abrasions, etc. Immediately report any problems or changes to your prosthetist.

Leave the liner inverted after removing it. Wash thoroughly using warm water and mild soap that does not irritate your skin. Do not use any abrasives/scrubbing pads on the liner. Rinse the liner thoroughly and place it, fabric side out, on the drying stand.

Failure to properly clean and rinse your liner may result in skin irritations or other fitting problems, and may also reduce the life of the liner.

Utilize two liners, and rotate use daily. This will allow for proper drying time and gel recovery, extending the life of your liners.

⚠ WARNING: Excessive exposure to products containing alcohol will damage the liner and void the warranty.

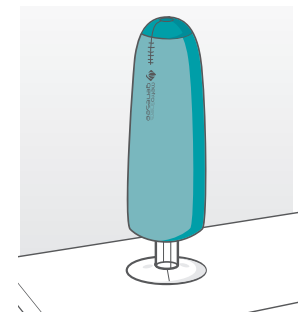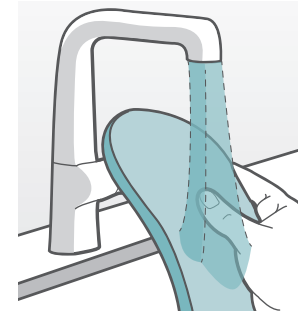

DRYING STAND ASSEMBLY



Assemble the drying stand as shown by inserting the tube through the disc, then onto the base. Finally, place the cap on top of the tube. The drying stand may be shortened by trimming the tube before placing onto the base.

WARRANTY/ RETURNS/ TRIAL PERIOD

Contact your prosthetist with any concerns regarding the warranty or trial period of the liner.