



# Partial Hand SOLUTIONS, LLC



## *User Guide* Titan Flex

## *Instructions for use*

- 1.** If using a silicone socket, apply a small amount of water to dampen limb and push on prosthetic socket to limb.
- 2.** To flex finger, flex wrist to apply tension on the lever and the finger will automatically flex. To extend finger back, flex wrist to disengage lock and manually extend back into position.
- 3.** New users should start with a wearing schedule and ease into a full day of use.
- 4.** To remove prosthetic socket from limb, pull on socket to remove or consult with your prosthetist or therapist on proper technique.
- 5.** Clean daily with fragrant free anti-bacterial wipes. Consult with your Doctor and prosthetist on preferred cleansing wipe.
- 6.** Prior to patient receiving the device, LOCTITE 242 should be applied to all screws.



*Shown connected to 3D  
printed prosthetic socket*

M-Fingers are distributed by:



**college park**  
800-728-7950 | 586-294-7950  
27955 College Park Drive, Warren, MI 48088



**Partial Hand  
SOLUTIONS, LLC**



**Partial Hand Solutions**  
86 Ladyslipper Lane  
Southington, CT USA 06489  
+1 (860) 538-5532



**EMERGO EUROPE**  
Prinsessegracht 20,  
2514 AP  
The Hague, Netherlands

**MADE IN THE USA**

988 INS UG PHS FLT 210602

